

BT-ACTB-L

ALS Cardiac Training Model : Lite

USER GUIDE



User Guide



English

Product Information

Product Name ALS Cardiac Training Model

Model Name BT-ACTB-L

Manufacturer BT Inc.

Country of Origin Republic of Korea

Certification



(Contains FCC ID : 2AVQ5BT-ACTB-L)

Product Specification

Dimensions 1720 X 540 X 240mm

Weight 22kg

Power Converter Input AC100-240V~, 50/60Hz, 2.5A Output DC 16.8V, 4A

Customer Service

Website www.btinc.co.kr

Address A-313, Samsung Techno-valley, 140, Tongil-ro,
Deogyang-Gu, Goyang-Si, Gyeonggi-do, 10594,
Republic of Korea

Telephone +82-2-2038-2903

E-mail bt@btinc.co.kr



Cautions for battery disposal

* Please follow the regulations of corresponding area for the disposal of used batteries.

Cautions

Please read it carefully to ensure correct usage and user's safety through understanding of the product.

- ① Do not use the product for other purposes described in the manual.
- ② Do not use the product for other purposes described in this manual.
- ③ If you disassemble or modify the product without any permission by BT Inc, you cannot get the warranty. Also, making imitated or altered products for commercial purposes are prohibited by law.
- ④ During the usage, please contact us after any electricity power off when the product is overheated or smelled something burning
- ⑤ Keep the product away form water or moisture. If the product gets wet, do not turn it on.
- ⑥ Do not use the product near strong magnetic fields or high voltage power lines.
The product might malfunction due to the electromagnetic waves.
- ⑦ Do not use the product under a weather of lightning. It can cause the product malfunction and increases the risk of electric shock
- ⑧ Do not use the product with temperatures below 0°C or above 40°C
(Operation temperature : 0°C ~ 40°C, Storage temperature : -10°C ~ 50°C)
- ⑨ Do not keep the product near hot places such as heaters. Deterioration of product or internal damage might occur
- ⑩ Never clean the product with chemicla substances such as solvent, benzene or thinner, as these may Cause ignition or cause the surface deterioration.
- ⑪ BT Inc. can change or adjust the product specification without any notice in order to improve the product.

Contents

1. Product Composition

1-1 Components	6
1-2 Consumables & Recommended Clinical Supply Sizes	7
1-3 How to store the product	8
1-4 Assemble/Disassemble the legs	9
1-5 Connect simulator	10
1-6 How to insert and replace battery	11

2. Function

2-1 Function description	12
2-2 Function description - Detailed	13

3. Checking O/S Specification

3-1 O/S specification	14
-----------------------	----

4. Checking H/W Specification

4-1 H/W specification	15
-----------------------	----

5. Install Application

5-1 Developer settings	16
5-2 Install app package file	17~18

6. Start Application

6-1 Select a product	19
6-2 Application settings	20
6-3 Connect Facilitator and Monitor	21~22
6-4 Connect Facilitator and Simulator	23~24

7. Facilitator Feature


7-1 Manual mode	25
7-2 Top menu	26
7-3 Bottom menu	27
7-4 Left menu	28
7-5 Right menu	29
7-6 Patient monitor settings	30
7-7 Patient monitor (Graph monitor)	31
7-8 Patient monitor (Numeric monitor)	32
7-9 Patient status viewer	33
7-10 Change the patient status	34
7-11 Scenario mode	35~42
7-12 Library	43
7-13 Result viewer	44~47

8. Monitor Feature


8-1 Patient monitor	48
8-2 Top menu	49
8-3 Bottom menu	50
8-4 Patient monitor settings	51
8-5 Patient monitor (Graph monitor)	52
8-6 Patient monitor (Numeric monitor)	53
8-7 Patient records	54
8-8 NIBP	55
8-9 Medications	56
8-10 Defibrillator	57
8-11 Pacer	58
8-12 Oxygen therapy	59
8-13 Patient status records	60

1. Product Composition


1-1. Components

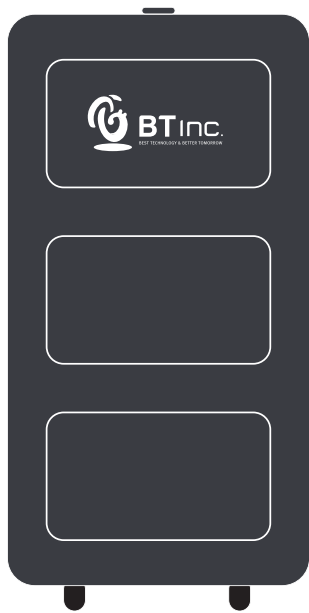

Storage case

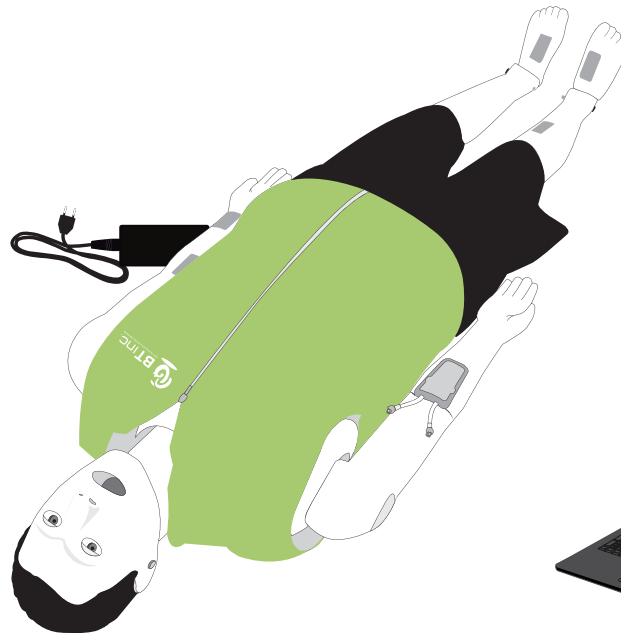


Full body simulator + Power adaptor

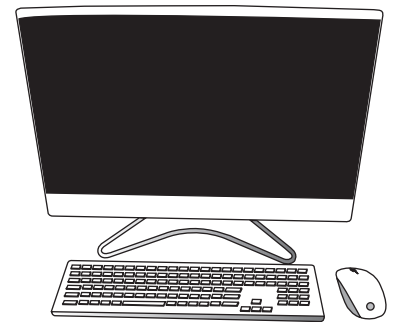

All-in-one PC for Monitor (Option)



Laptop for Facilitator (Option)

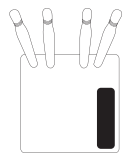


Wifi router (Option)




Defibrillation electrode cover for pads




Defibrillation electrode cover for leadwires

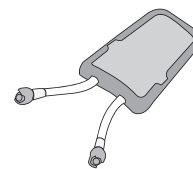


IV pad

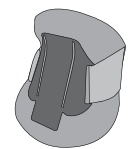

Cervical collar

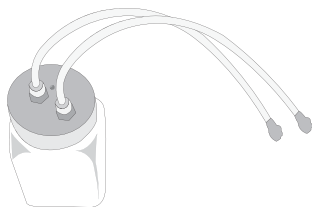

Bottle&tube for IV

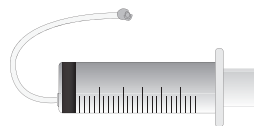

Syringe for cleaning



Simulated blood



Lubricant

*Etc – Clothing jacket/pants, Ankle/Knee protector, Battery extension cord, Silicon pad for electrode reinforcement



1-2. Consumables & Recommended Clinical Supply Sizes

Consumables	
<ul style="list-style-type: none"> • IV pad • Skin(face, torso, head) 	<ul style="list-style-type: none"> • Lubricant • Simulated blood

Recommended Clinical Supply Sizes	
Needle	3cc~5cc, 22G~24G
ETT	I.D. 7.0~8.0mm
I-gel / LMA	#3, #4
Combi-tube	37Fr
OPA	88mm(Green)
NPA	6~7(24~28Fr)





- The following clinical supply sizes are recommended for use with ACTB-L model.
- Other sizes may cause damage to the products.
- Lifetime of the consumables may differ depending on the skill level of the users.

1-3. How to store the product

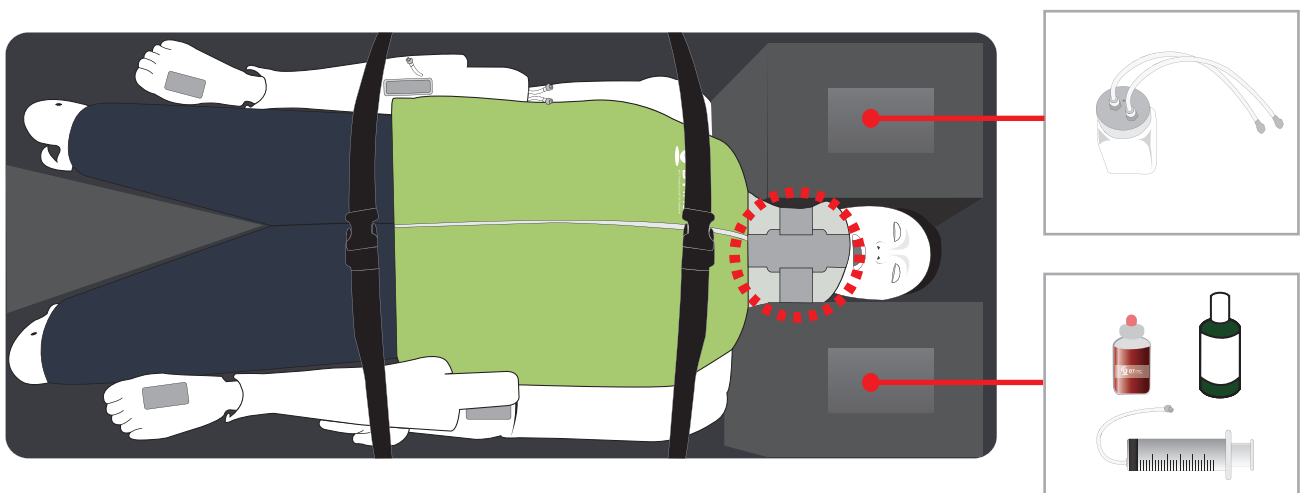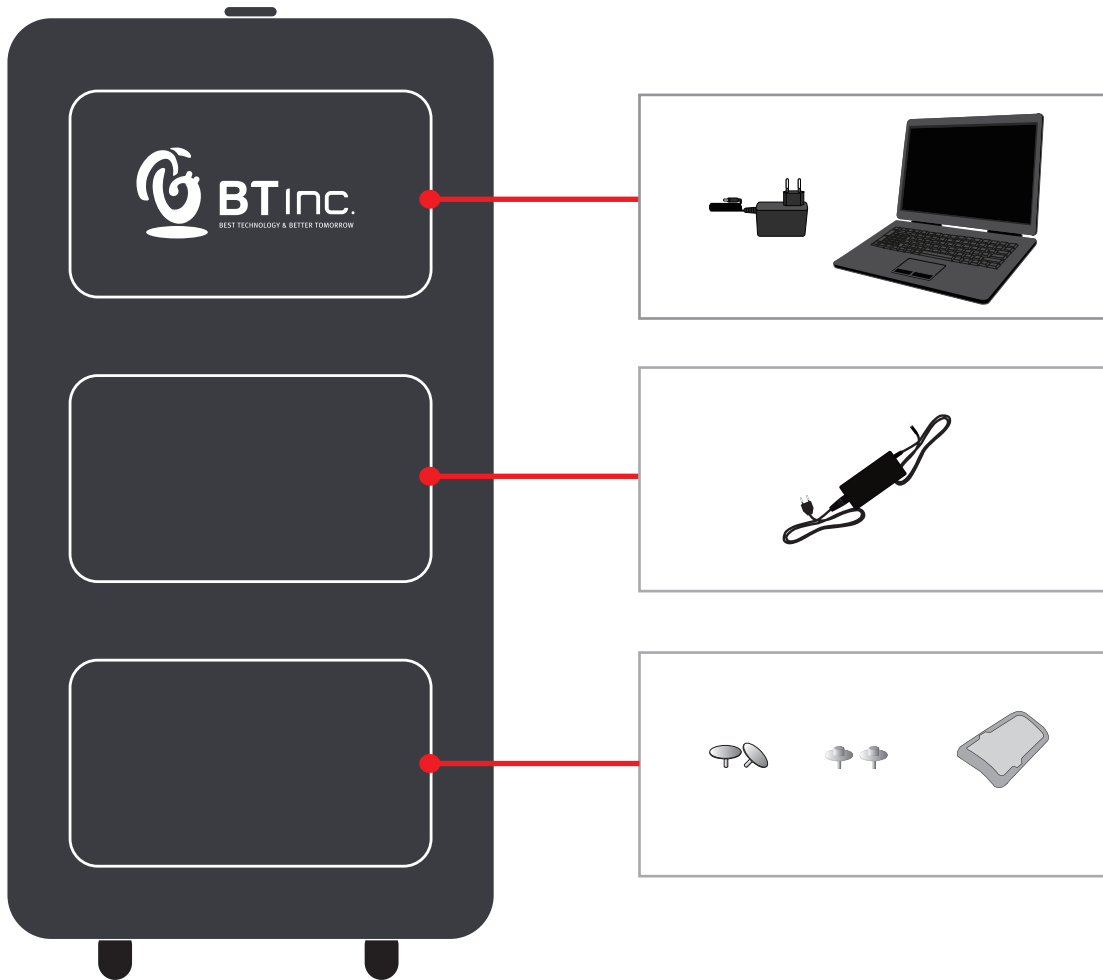


※ Make sure to wear a neck protector on the simulator during storage or transportation.

1-4. Assemble/Disassemble the legs

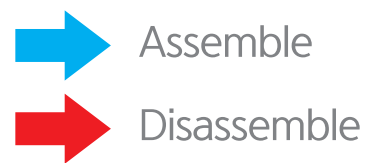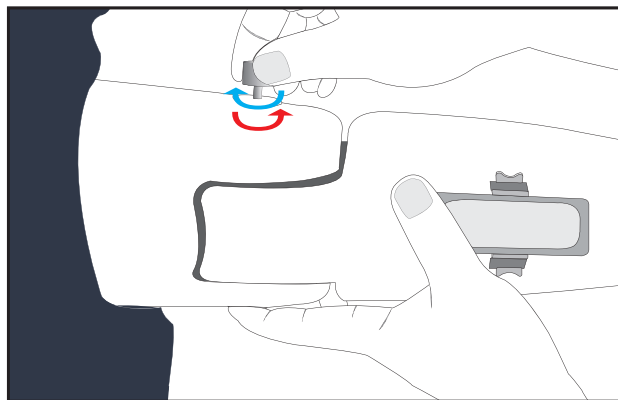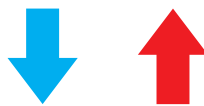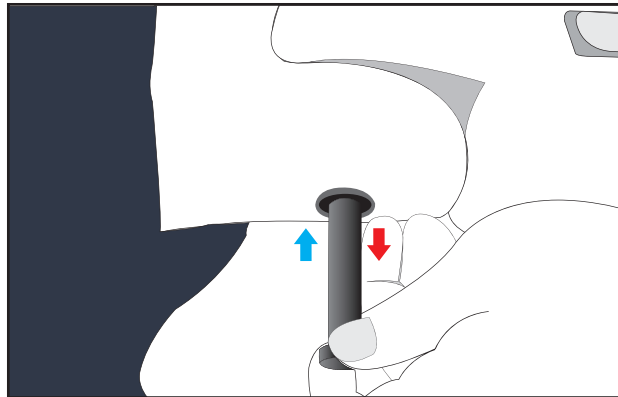


※ Disassembly is the reverse order of assembly.


1-5. Connect simulator

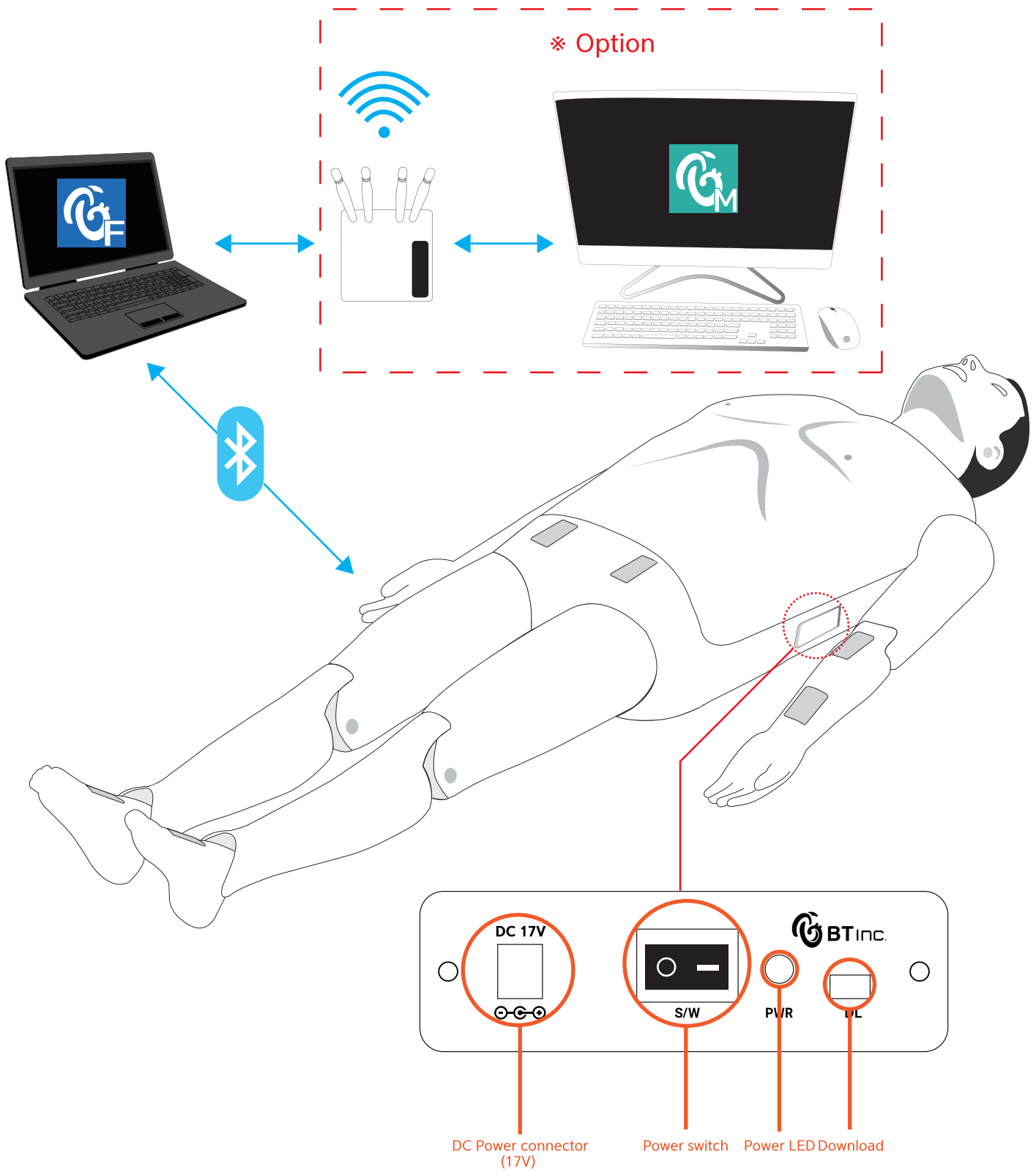


1-6. How to insert and replace battery

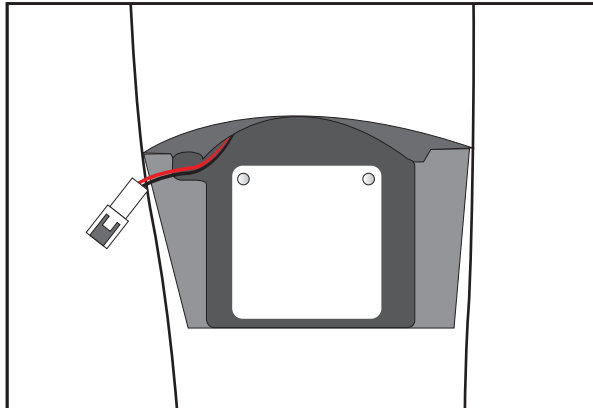

1. Open the battery cover.

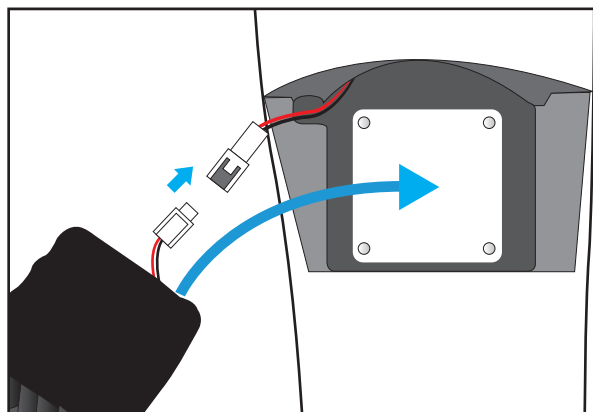

2. Connect the connectors and insert the battery into the simulator.

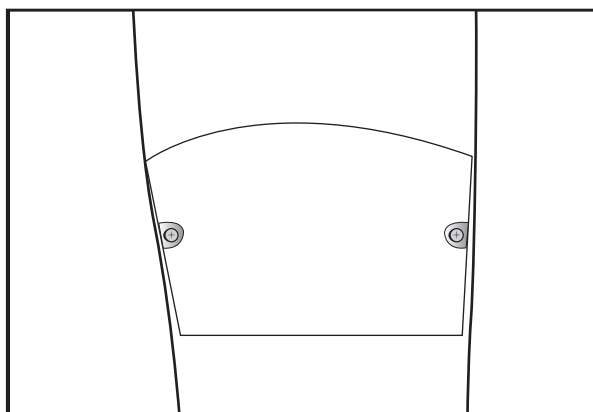


3. Close the battery cover.

※ The battery is located on the right thigh.

2. Functions

2-1. Functions description

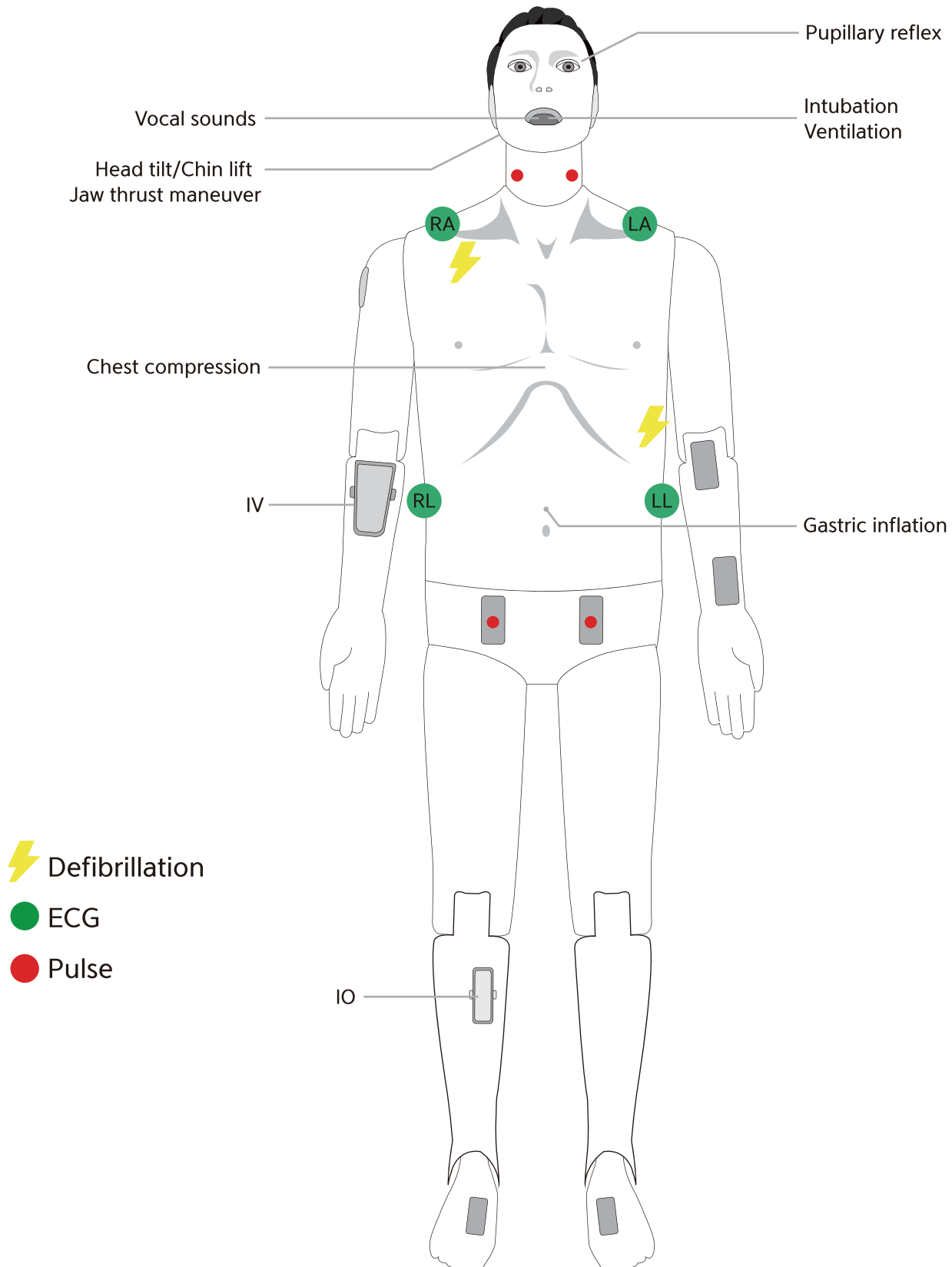

2-2. Function description – Detailed

🟡 Eyes

- ① Light reflex / Consensual light reflex
- ② Adjust pupil size
 - 1~8 mm
- ③ Adjust light reflex speed
 - Prompt / Slow
- ④ Eyelids : Open / Closed (manually)

🟡 Airway management

- ① Head tilt / Chin lift
- ② Jaw thrust
- ③ Bag-Valve-Mask ventilation
- ④ Oral / Nasal intubation
 - ETT / LMA / I-gel / Combi-tube
 - Detect Normal / Right mainstem intubation
- ⑤ Detect esophagus Intubation and gastric inflation
- ⑥ Measure force on incisors
- ⑦ Airway obstruction

🟡 Circulation

- ① 3-4 Lead ECG
- ② Defibrillation and cardioversion
 - Defibrillation waveform
 - CPR waveform
- ③ Pulse
 - Pulse sites
 - : Left / Right carotid artery
 - : Left / Right femoral artery
 - Pulse intensity
 - : Strong / Moderate / Weak
- ④ ABP waveform (Software)
- ⑤ PAP waveform (Software)

🟡 Breathing

- ① Spontaneous breathing
 - Unilateral and bilateral chest rise
 - Normal breathing sound
- ② SPO2 waveform (Software)
- ③ ETCO2 waveform (Software)

🟡 CPR

- ① Chest compression
 - Compression depth
 - Hands placement
(Correct/Upper/Lower/Left/Right)
 - Compression rate
 - Incomplete recoil
 - Hands-off time
- ② Artificial respiration
 - Ventilation volume
 - Inspiratory time
 - Ventilation per minute

🟡 Vascular access

- ① IV (Right arm)
- ② Medication (Software)

🟡 Speech


- ① Vocal sounds
 - 10 cases

3. Checking O/S Specification

3-1. O/S specification

Minimum System requirements : **Windows 10, Version 1809 or later**

- ① Search **winver** at the search box next to windows start menu icon.

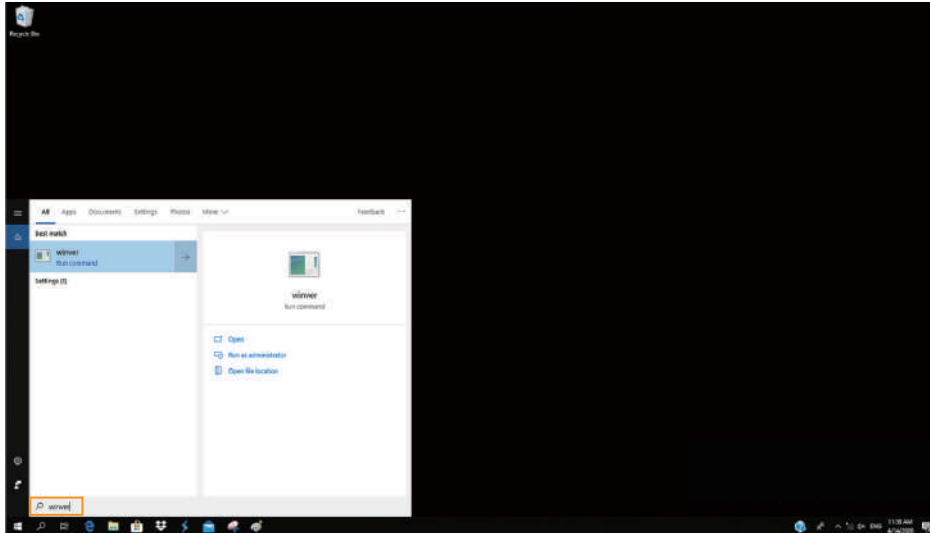

- ② Window version can be checked at the pop-up screen.




4. Checking H/W Specification


4-1. H/W specification

- Resolution : 1920 X 1080
- Text Size : 100%

① Search **Display settings** at the search box next to windows start menu icon.

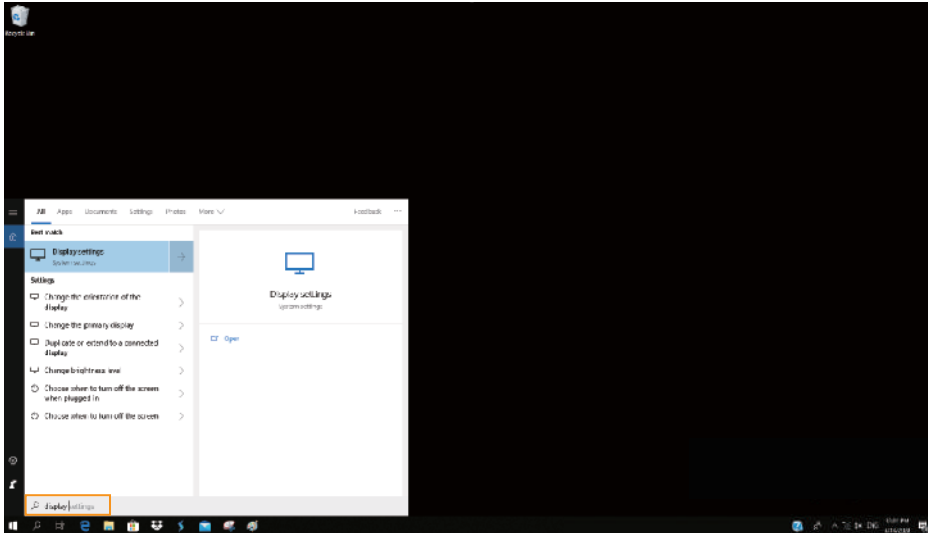


② Set the **Change the size of text** and **Resolution** as below.

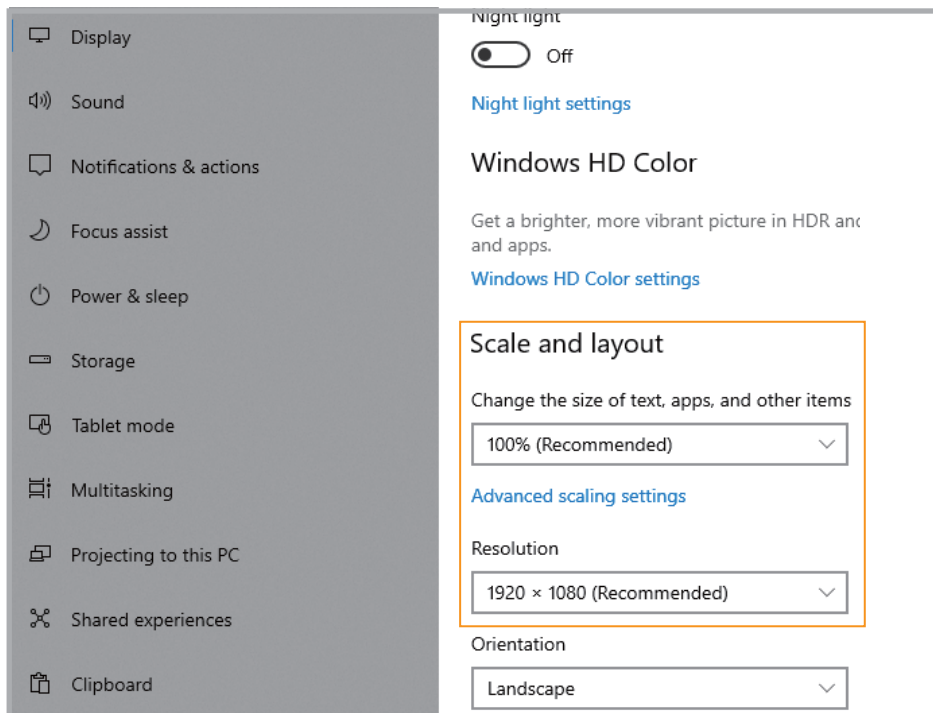


5. Install Application

5-1. Developer settings

- ① Search **Developer settings** at the search box next to windows start menu icon.

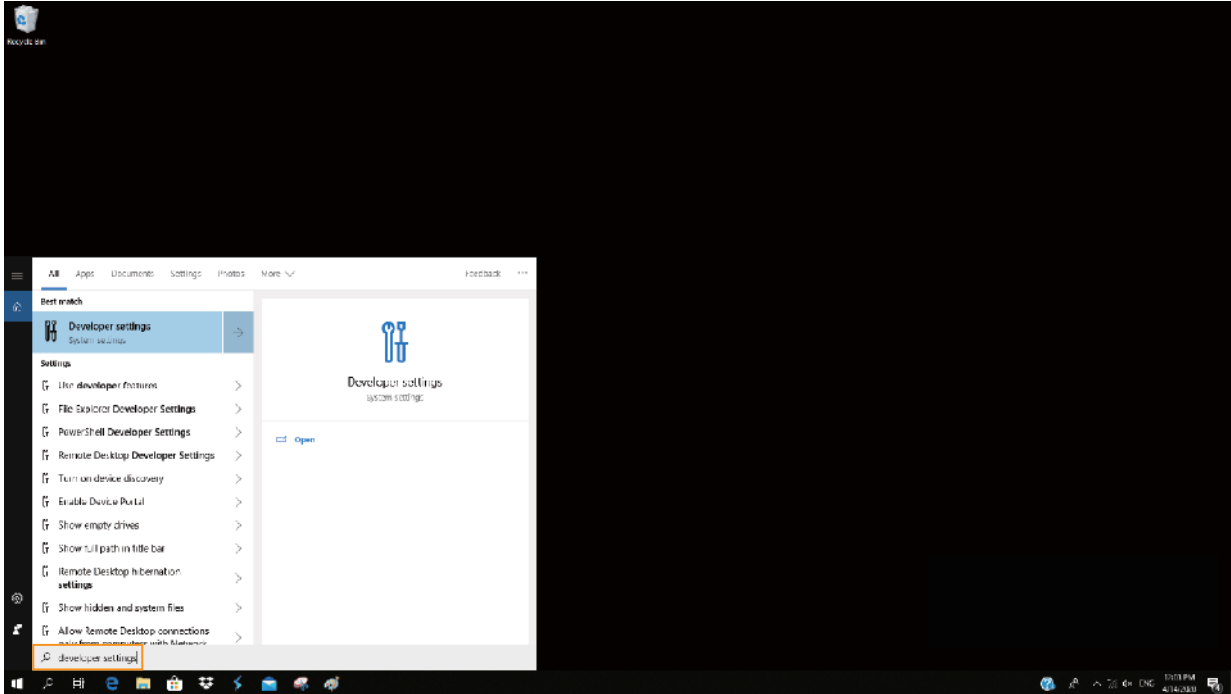


- ② Activate **Developer mode** at the screen.

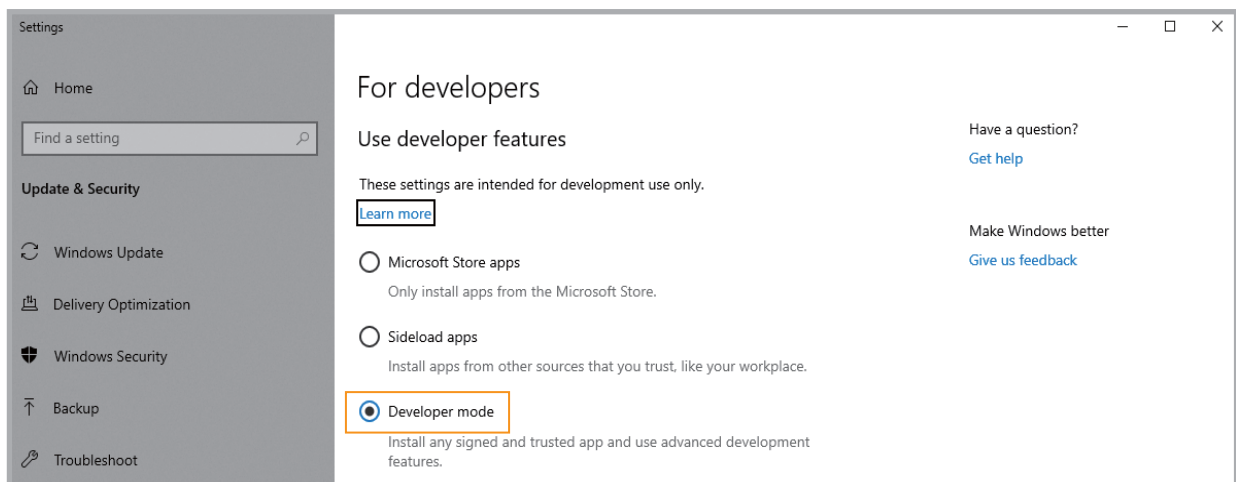


5-2. Install app package file


- ① Select the app package file and right-click to launch **PowerShell**.

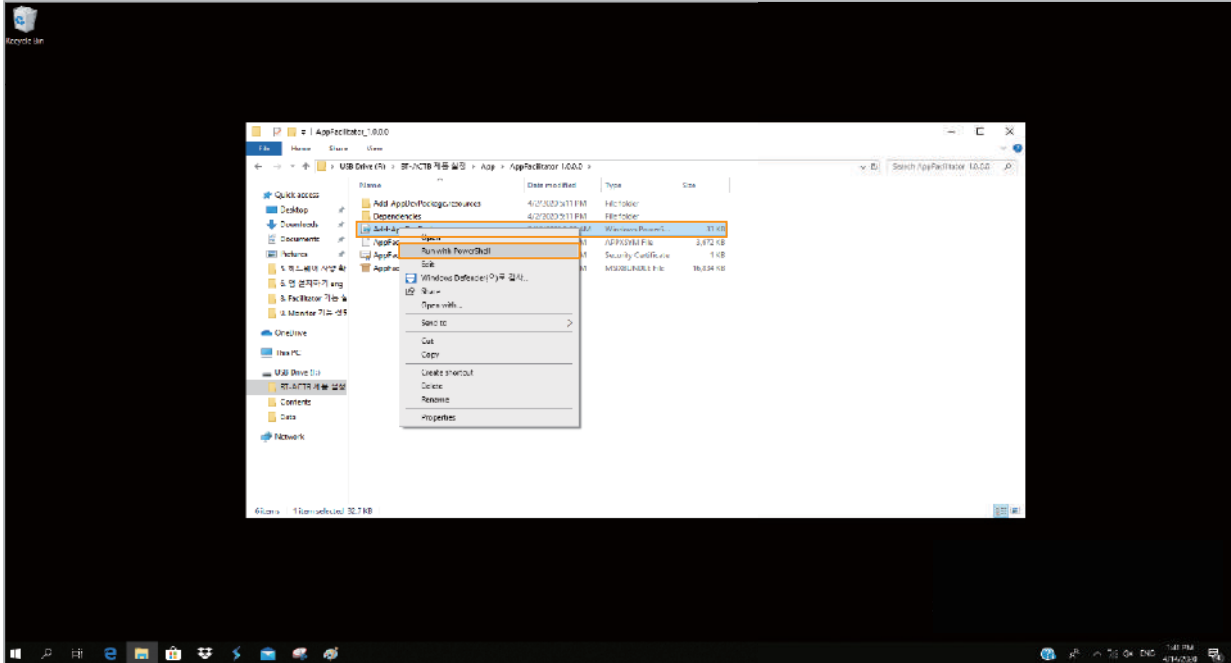


- ② Install the app by following instructions.
 - It may take some time depending on the PC's specification.

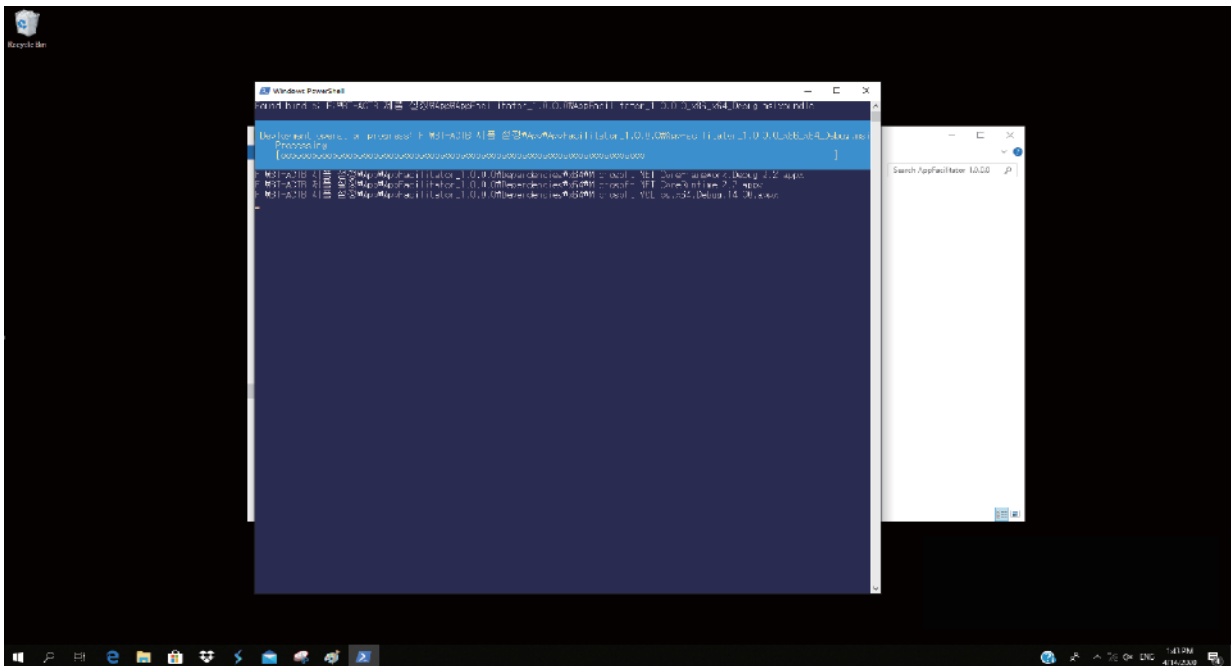

③ When the installation is completed, the app will be on the **windows start menu**.

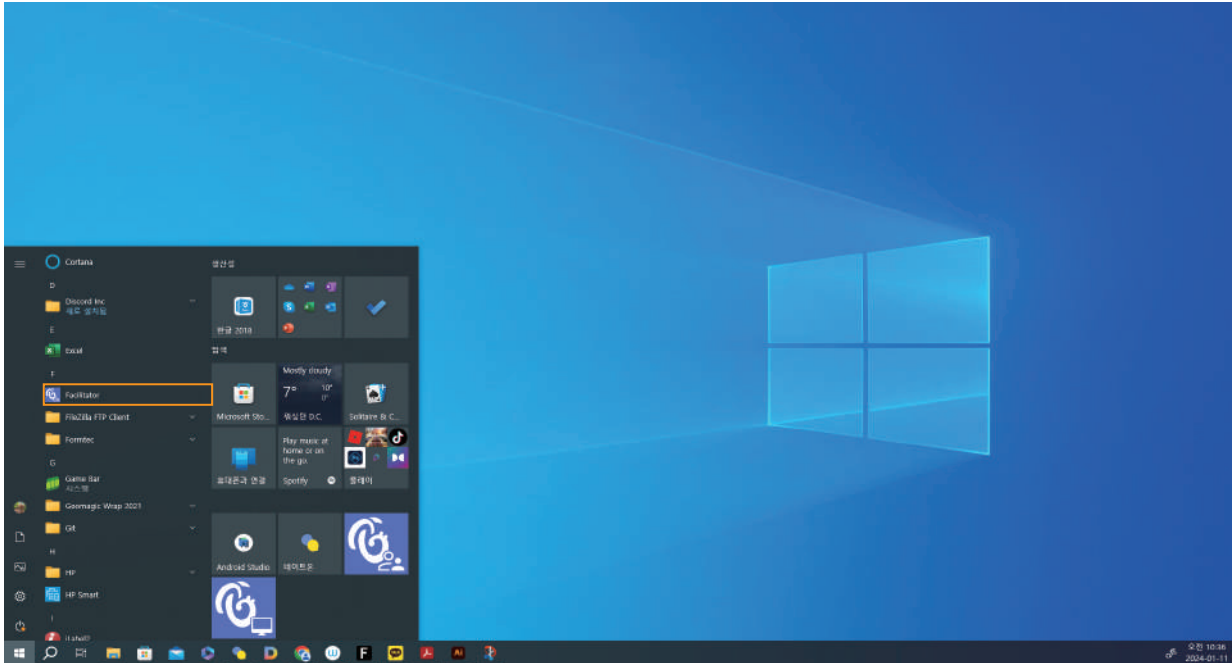


④ The app can be moved to the desktop screen by dragging the mouse.

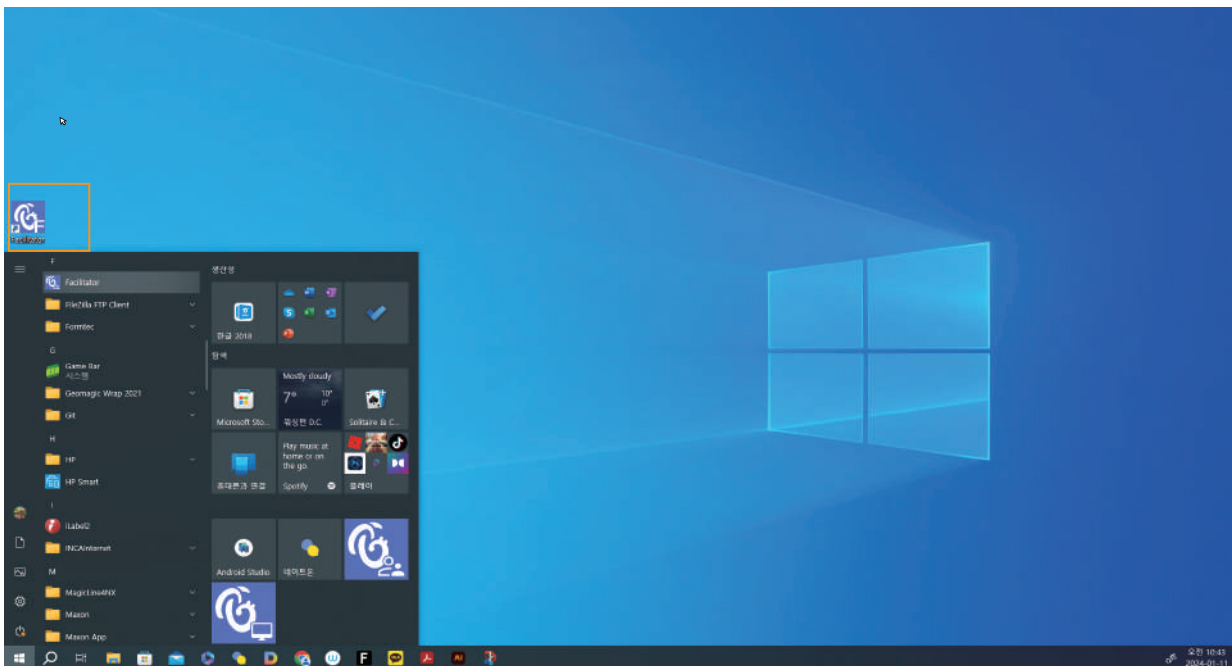


6. Start Application

6-1. Select a product

- ① Select the product.

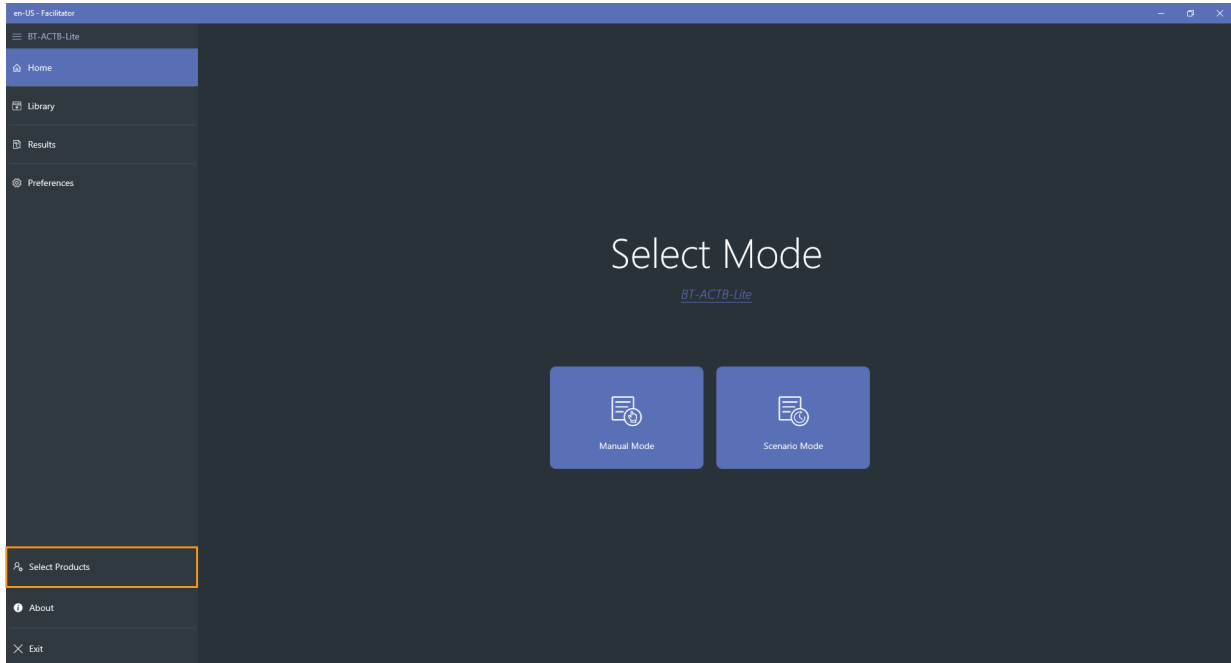


- ② The selected software is started.

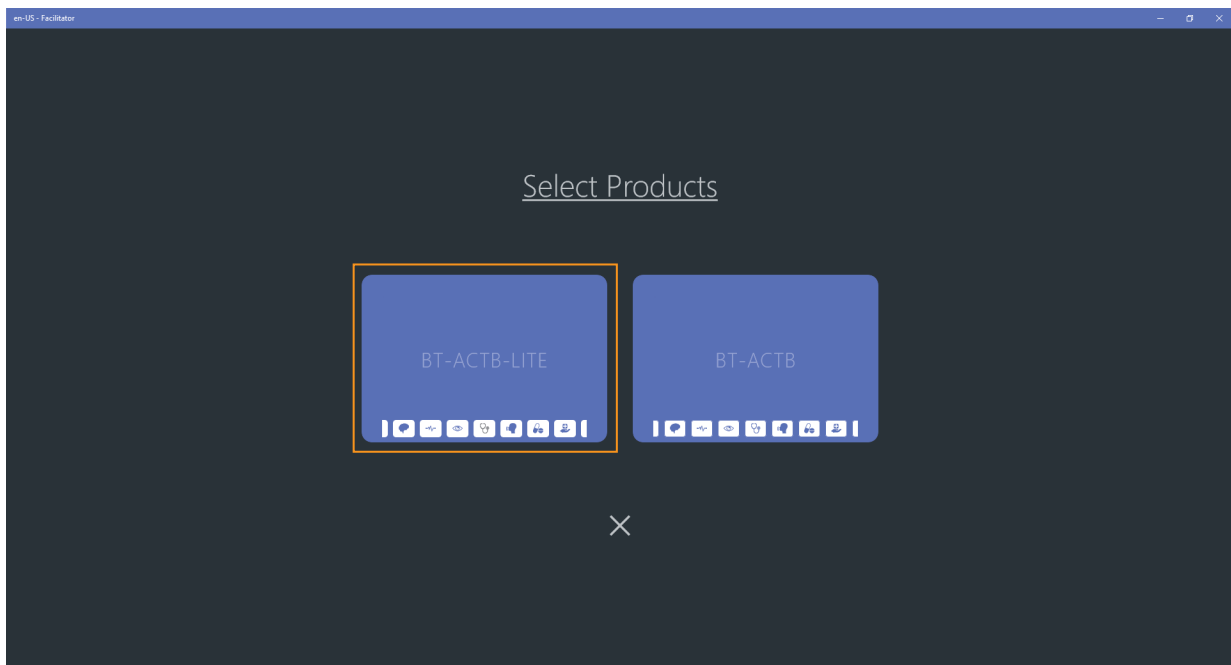


6-2. Application settings

- ① Start the app and select **Preferences menu** at the left side.

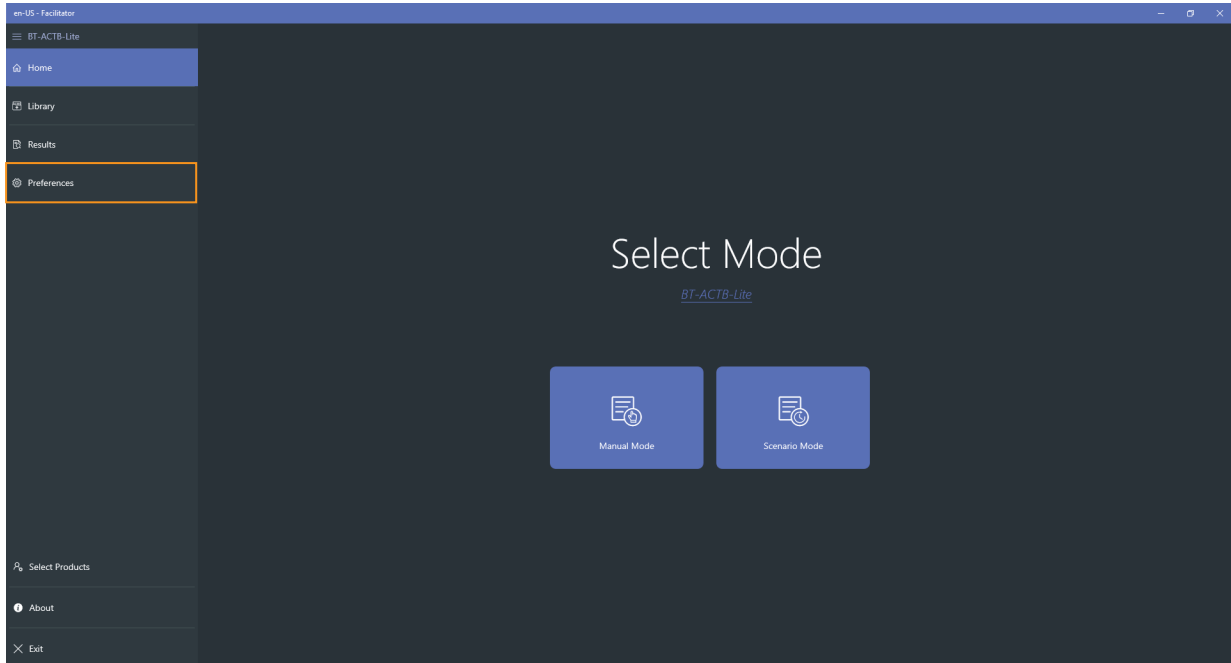

< Home screen >

- ② Select the file system from the **App permissions** at the right side of the screen.

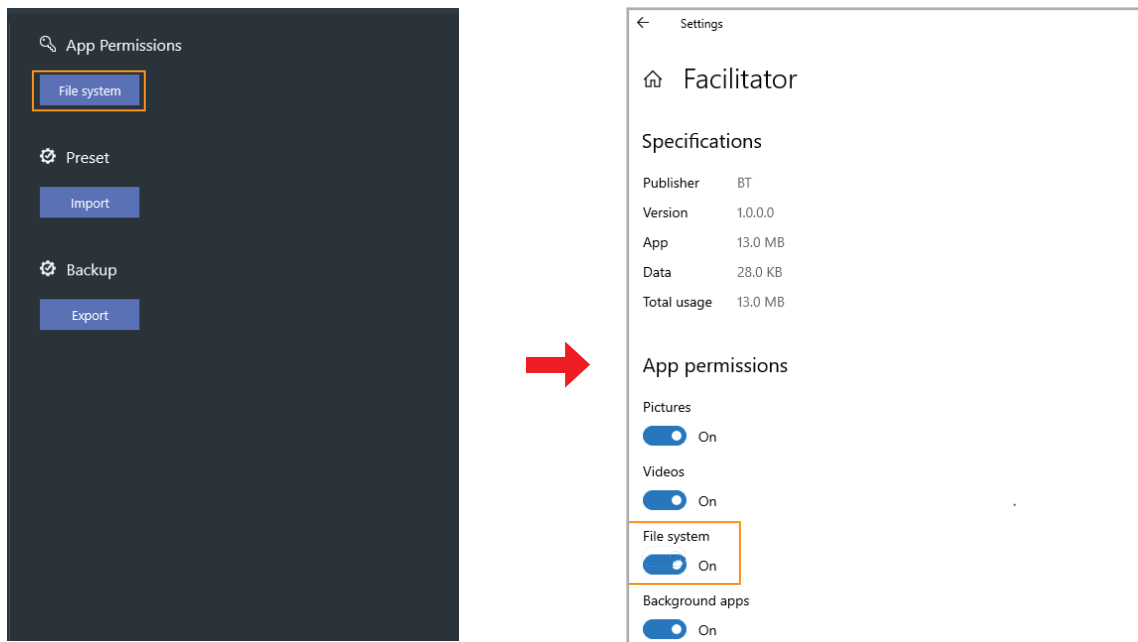


- ③ Activate the file system at the app permissions screen.
- ④ The apps will be automatically shut down when the setting is completed.

6-3. Connct Facilitator and Monitor

① Connect Wi-Fi

– Connect **Facilitator** and **Monitor** to the offered Wi-Fi router.

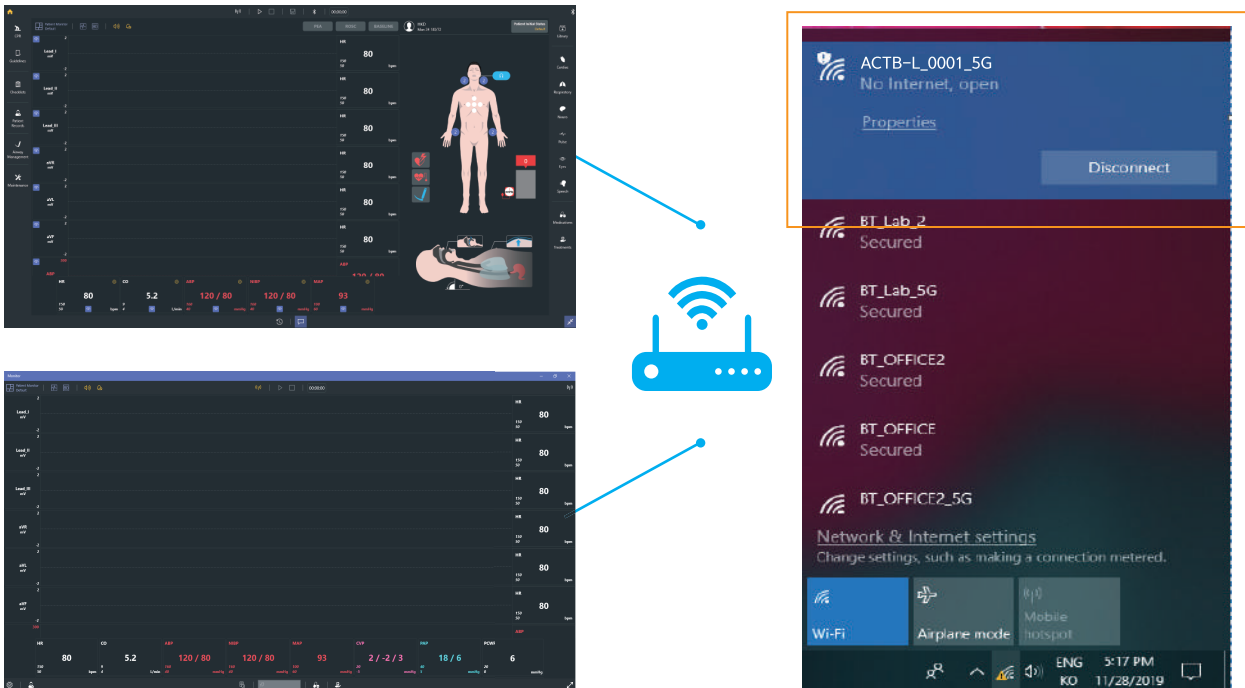

② Facilitator Settings

– Click the **Start button** at the top menu.




③ Monitor Settings

– Click the **Wi-Fi button** at the top menu.




④ When the two apps are connected, you can see the message as shown below.

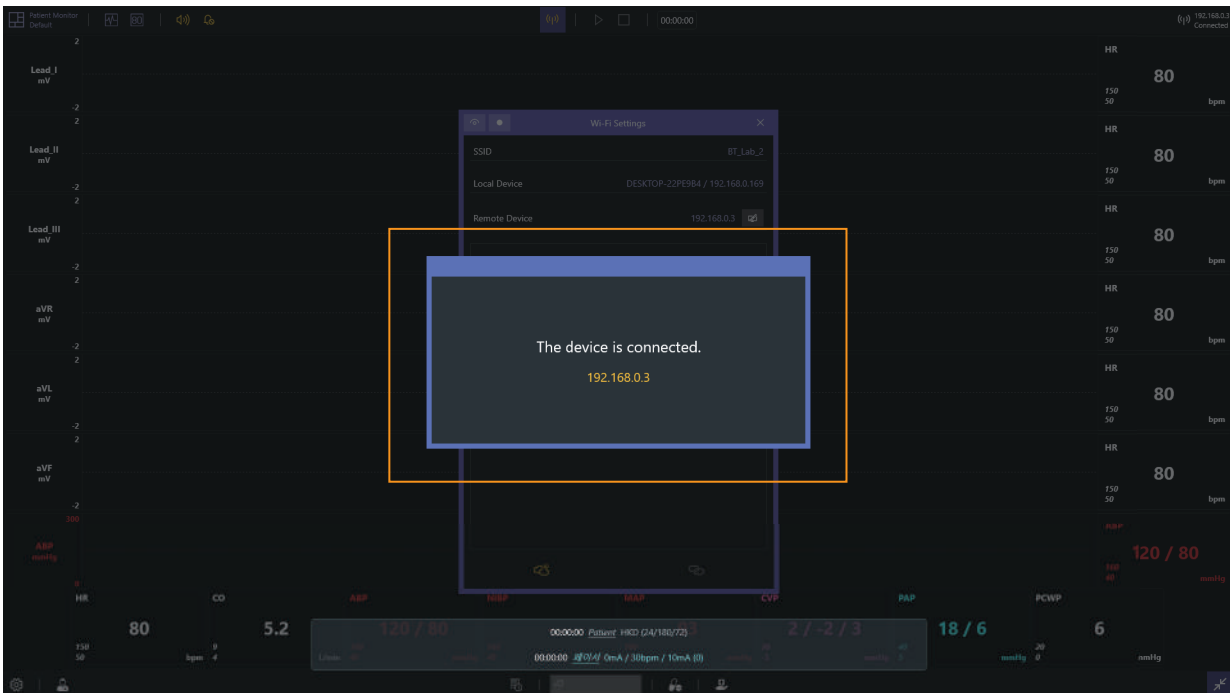


6-4. Connect Facilitator and Simulator


- ① Turn on the simulator.
- ② Click the **Bluetooth button** at the top menu.

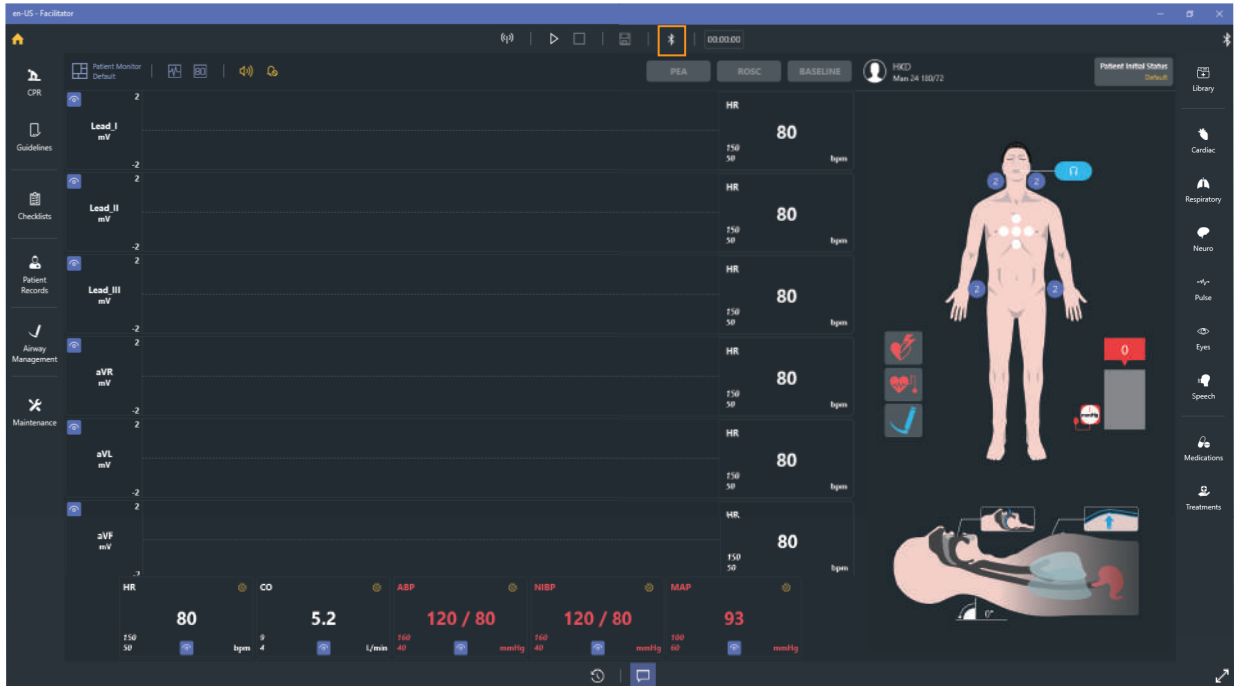

- ③ Find the **simulator's serial number** from Bluetooth list and connect it with the simulator.

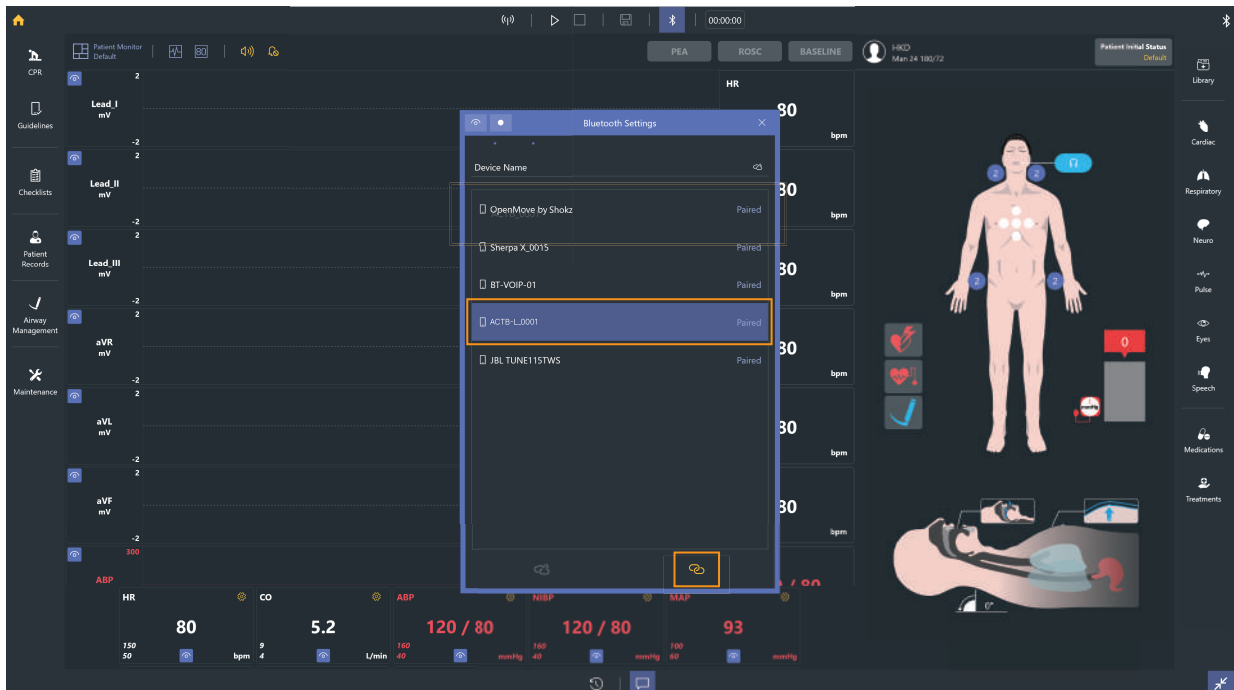

④ The Bluetooth connection status can be checked in the upper-right corner.

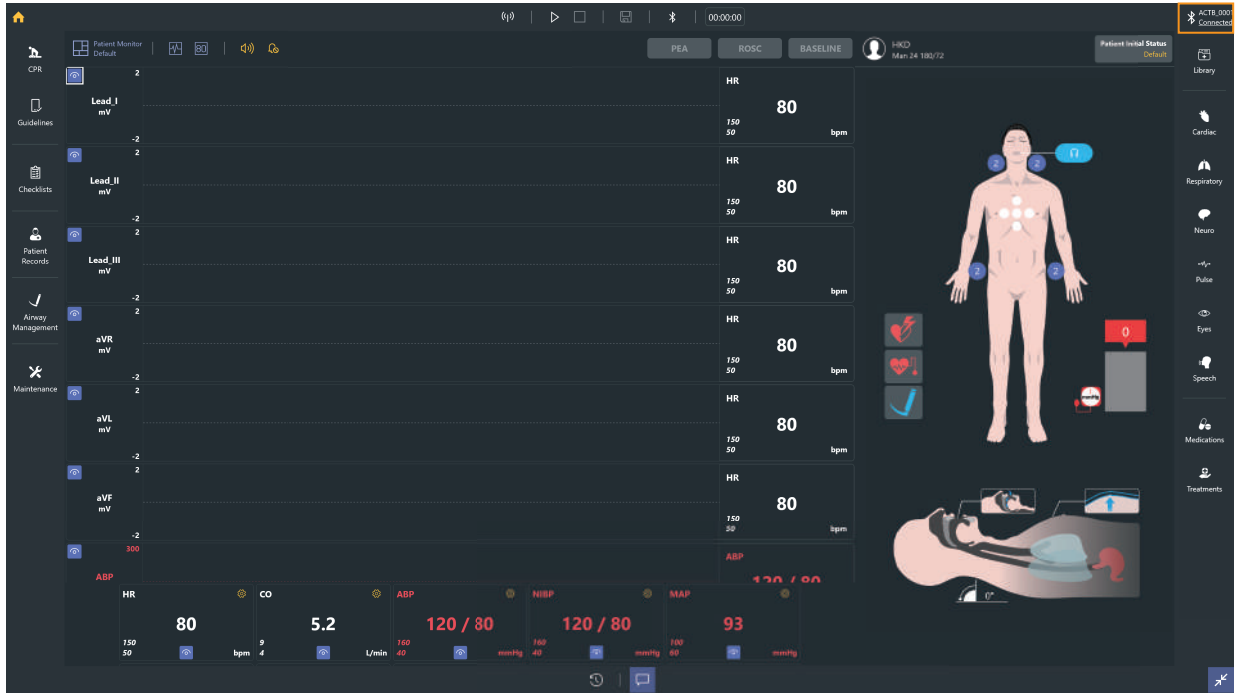


✳ If the simulator is not paired to the laptop, the pairing operation is completed automatically.

7. Facilitator Feature

7-1. Manual mode

① Select **Manual mode** from the home screen.

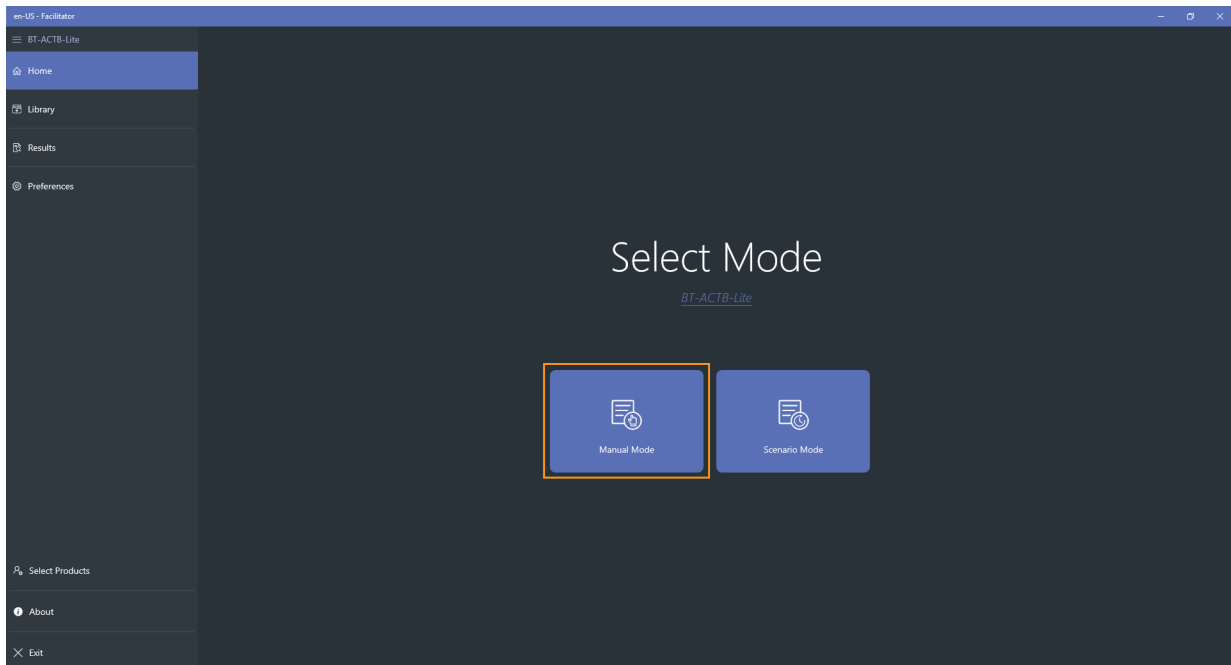










② Layout of manual mode

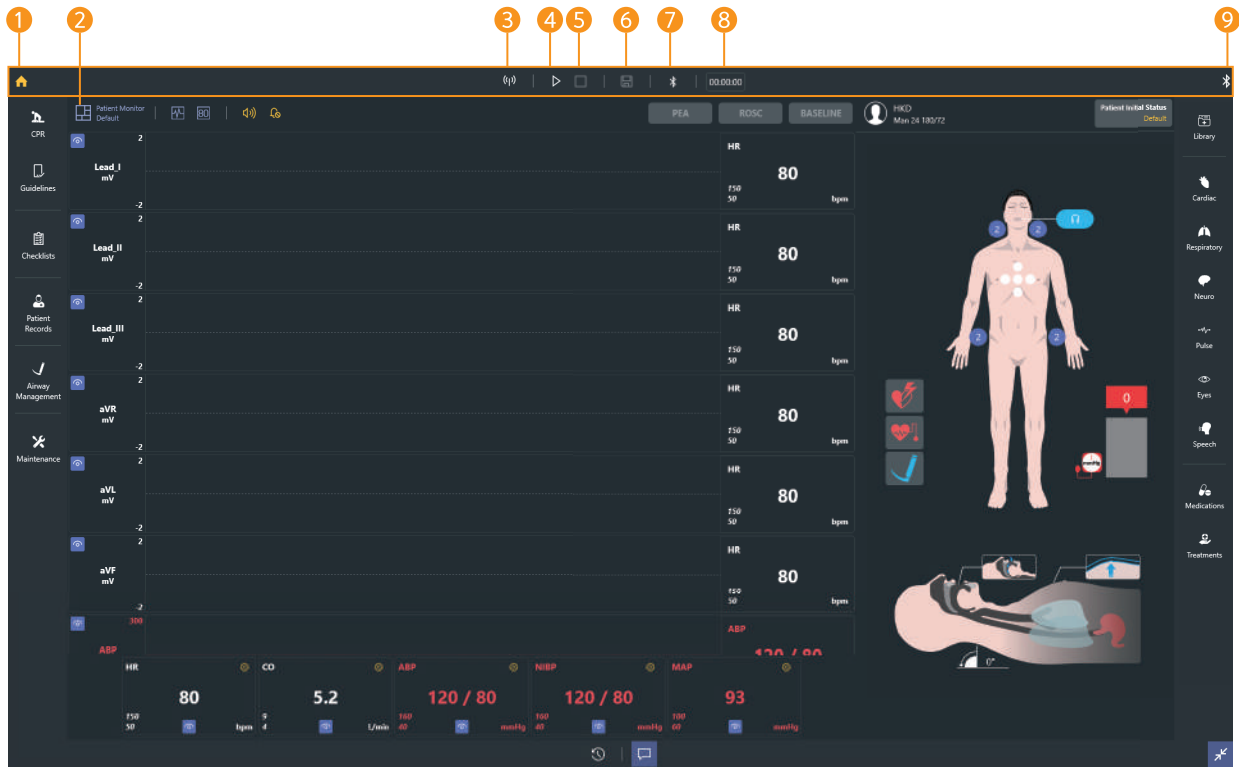

- ① Top menu
- ② Bottom menu
- ③ Left menu
- ④ Right menu
- ⑤ Patient monitor
- ⑥ Patient status viewer

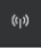7-2. Top menu



- 1 **Home**  : Go to the home screen.
- 2 **Patient monitor settings**  : Change settings of the layout of patient monitor and the audio.
- 3 **Wi-Fi settings**  : Connect with Monitor.
- 4 **Start training**  : Start and pause the training.
- 5 **End training**  : End the training.
- 6 **Save results**  : Save the training result.
- 7 **Bluetooth settings**  : Connect Facilitator to Simulator.
- 8 **Training time**  : Display the training time.
- 9 **Bluetooth connection status** : Displays the simulator connection status.


7-3. Bottom menu



1 Event Logs

- Record changes to parameters, medications, treatments, checklists, and simulators.
- You can select and view the items you want by using the filter.


Time	Parameter	Value	Unit
00:00:00	Patient Status	Patient Initial Status	1
00:00:02	Airway Management	Jaw Thrust	2
00:00:02	Airway Management	Esophageal Intubation (ET)	3
00:00:02	Heart Rate	80bpm → 111bpm	4
00:00:06	Arterial Blood Pressure	120/80 mmHg → 87/60 mmHg	5
00:00:06	Cardiac Rhythm	NSR → VF (Mega)	6
00:00:14	Defibrillator	Biphasic: Defibrillation ▶ 150J (1)	7
00:00:20	Eyes	Right ▶ Inactive (3mm) Left ▶ Inactive (3mm)	8
00:00:27	Fraction of Inspired Oxygen	FiO2 ▶ 10%	9
00:00:28	Nasal Cannula	Flow Rate ▶ 2LPM	10

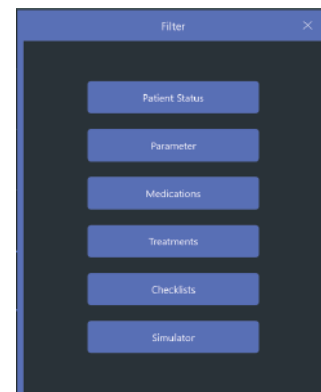

2 Pop-up message : You can show or hide the pop-up message.


3 Full-screen mode : You can switch to full screen mode.

7-4. Left menu

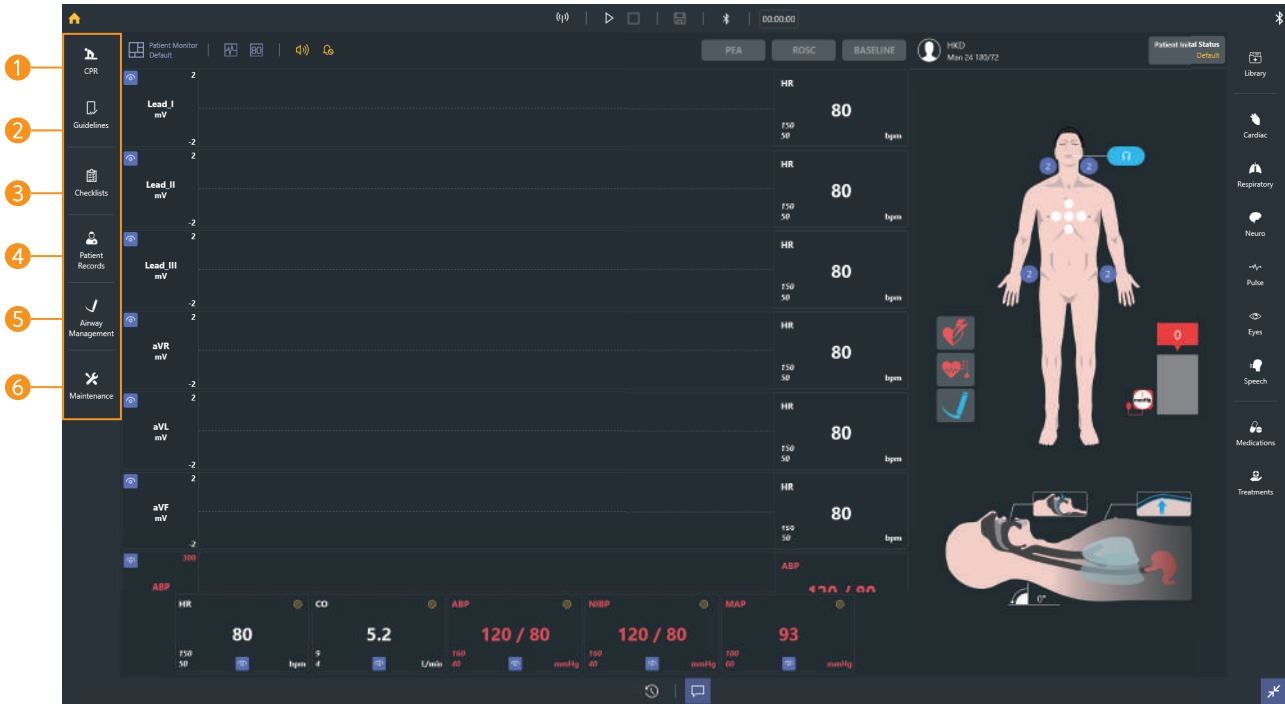


- 1 **CPR** : Show the CPR feedback in real-time with graph and chart.
- 2 **Guidelines** : You can set CPR guidelines.
- 3 **Checklists** : You can create the checklist for the training.
- 4 **Patient Records**
 - Set or check the patient profile.
 - Send the patient history and diagnostic result.
 - Set or check the initial patient status.
- 5 **Airway management** : You can choose the intubation device or set the force on incisors.
- 6 **Maintenance** : You can check the simulator's pneumatic operation status or calibrate the sensor values.

7-5. Right menu

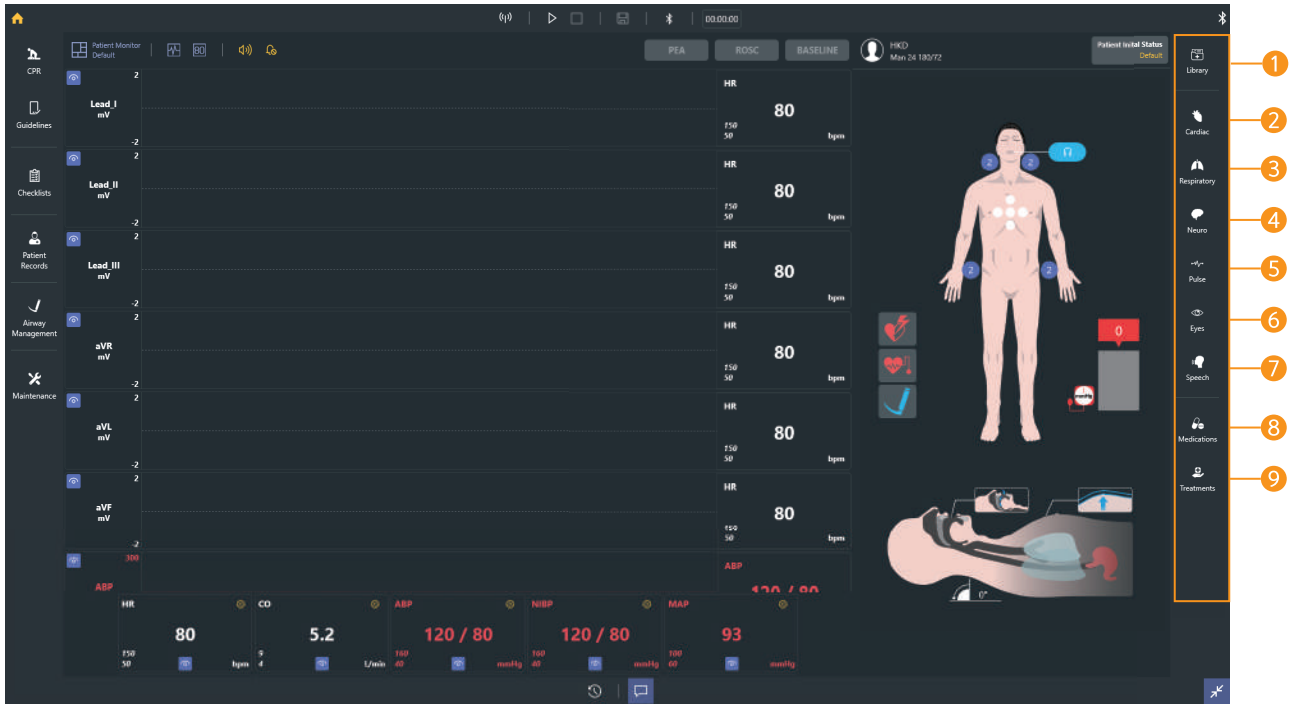


- 1 **Library** : You can set and save patient parameters at once.
- 2 **Cardiac** : You can set cardiac parameters such as cardiac rhythm, heart rate, cardiac output.
- 3 **Respiratory** : You can set respiratory parameters such as chest rising, airway obstruction, respiratory rate.
- 4 **Neuro** : You can set nervous parameters such as ICP, body temperature.
- 5 **Pulse** : You can adjust pulse status and the intensity.
- 6 **Eyes** : You can set the pupil status.
- 7 **Speech** : You can set the speech and the breathing sound.
- 8 **Medications** : Create a list of medication needed for the training and use the medication to make a history log or change the patient status in Scenario mode.
- 9 **Treatments** : You can train defibrillation, pacing, oxygen therapy.

7-6. Patient monitor settings

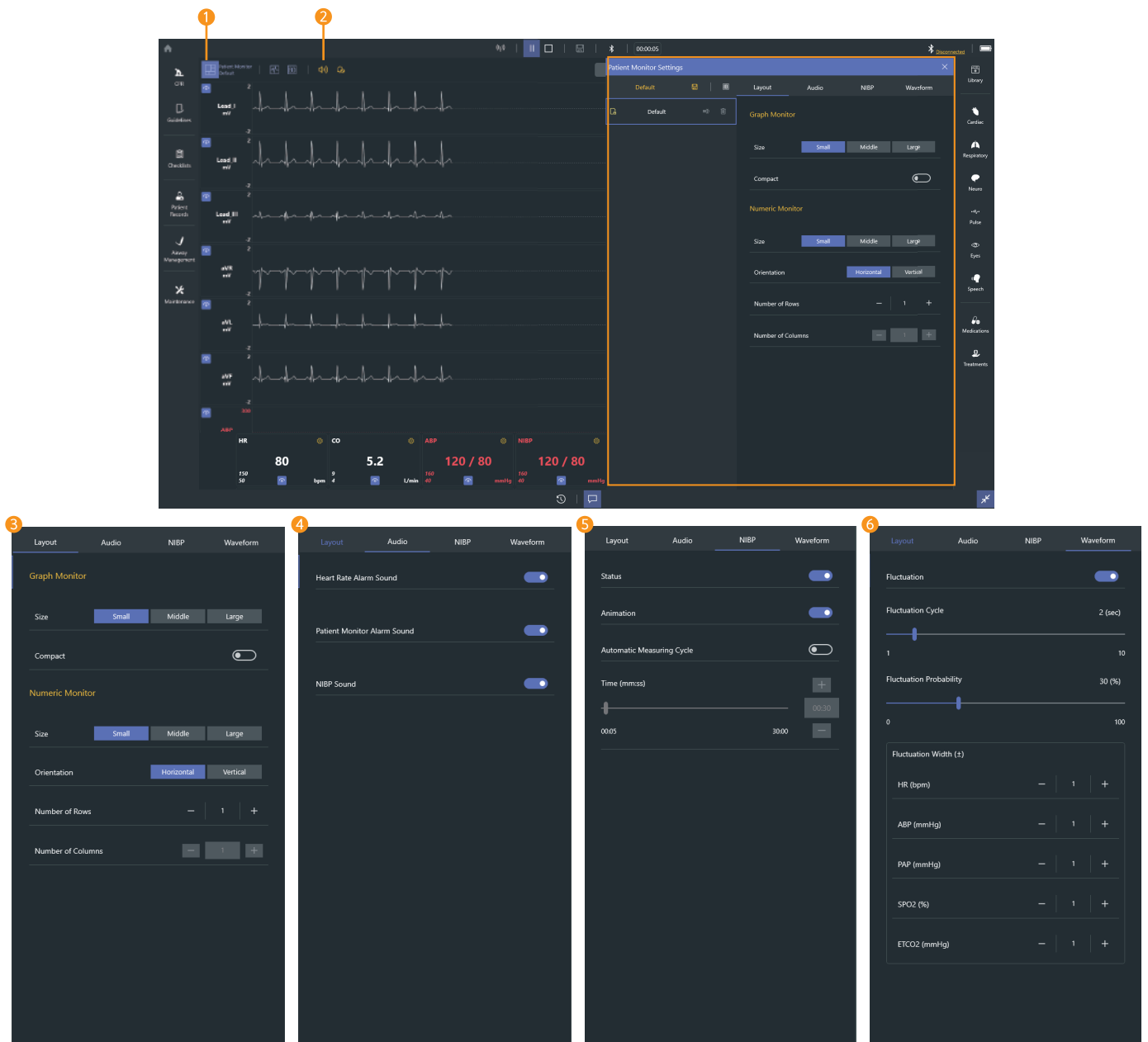

- 1 Patient monitor settings
 - You can select a saved patient monitor or create a new one.
- 2 Mute
 - Turn the sound ON/OFF.
- 3 Patient monitor layout settings
 - You can set the layout of the graph monitor and numeric monitor.
- 4 Audio settings
 - You can set the heart rate alarm, patient monitor alarm, and NIBP sound.
- 5 NIBP settings
 - You can set the NIBP auto cycling.
- 6 Waveform settings
 - The fluctuation for the graph of patient parameter can be set.

7-7. Patient monitor (Graph monitor)

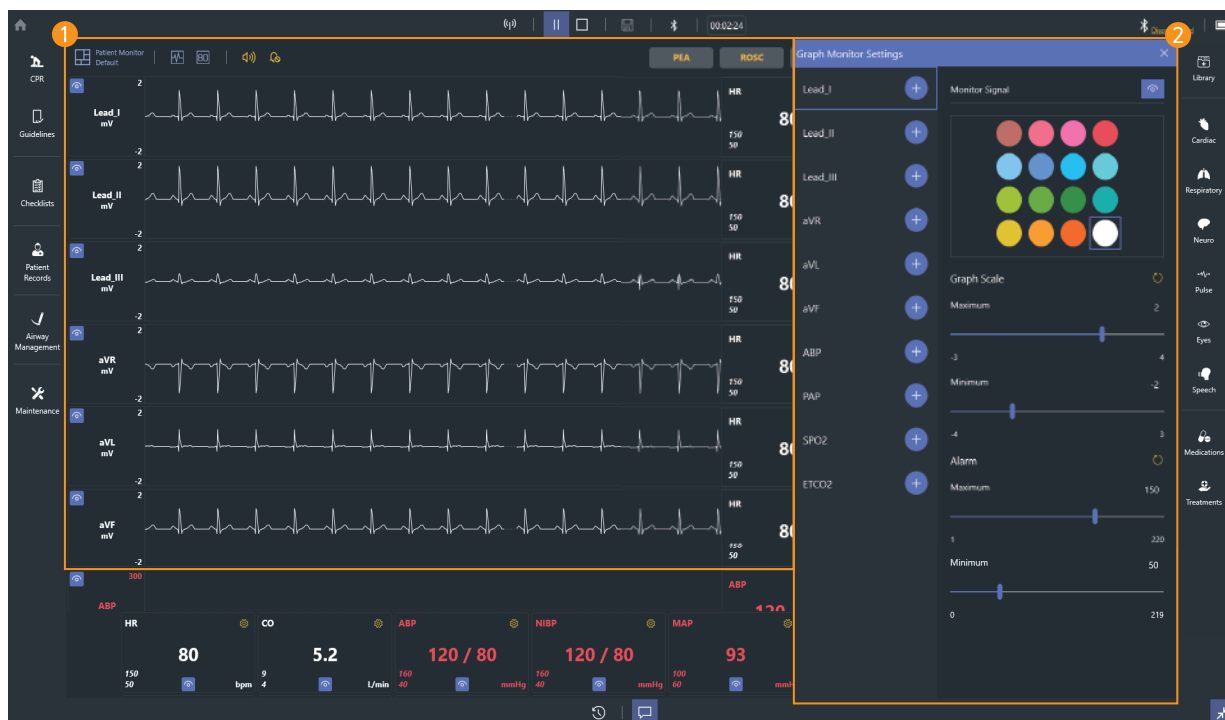

1 Graph monitor

- Display ECG, ABP, PAP, SPO2, ETCO2 waveform.
- You can change the ECG waveform.

2 Graph monitor settings

- You can add or delete waveforms to the graph monitor.
- You can change the color of the waveform.
- You can display or block the signal of the waveform.
- You can adjust the graph scale.
- You can set the alarm.

7–8. Patient monitor (Numeric monitor)




1 Numeric monitor

- Display numeric values such as HR, CO, ABP.

2 Numeric monitor settings

- You can add or delete patient parameters in the numeric monitor.
- You can change the color of the patient parameters.
- You can show or hide patient parameters.
- You can set the alarm.





7–9. Patient status viewer

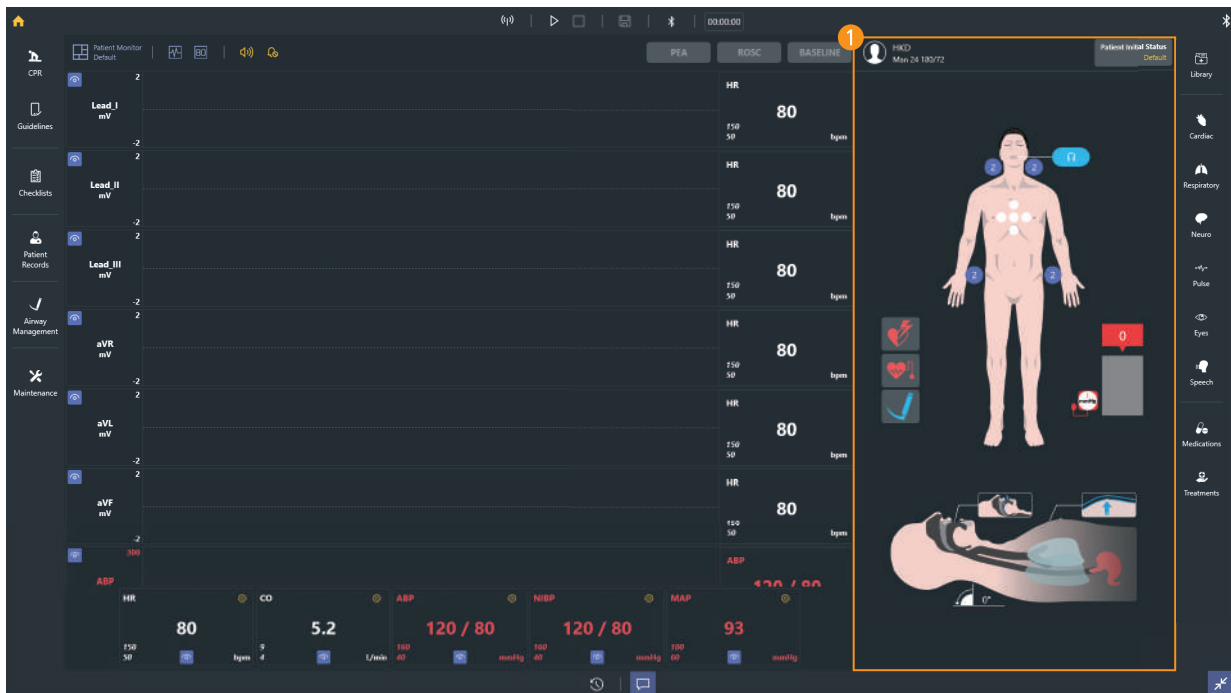


1 Display or change the current patient status.



- Display the patient profile.
- Display the hands placement during chest compression.
- Display the pulse intensity and the position of palpation.
- Display the airway angle.
- Display the jaw thrust maneuver.
- Display the force on incisor.
- Display the position of intubation.
- Display the stomach inflation.
- Display the chest rising and the airway obstruction.

7-10. Change the patient status

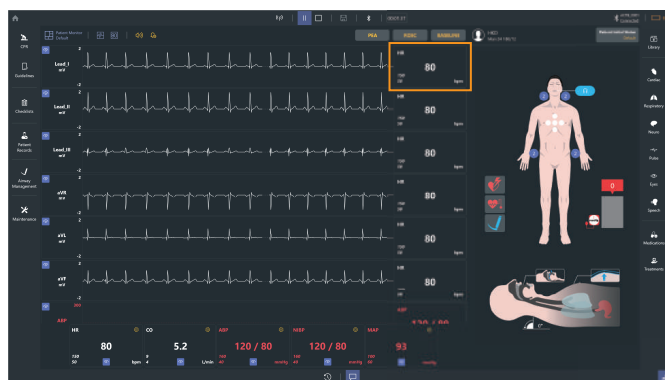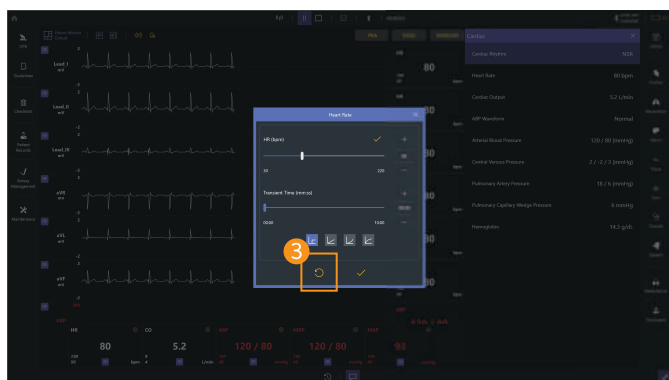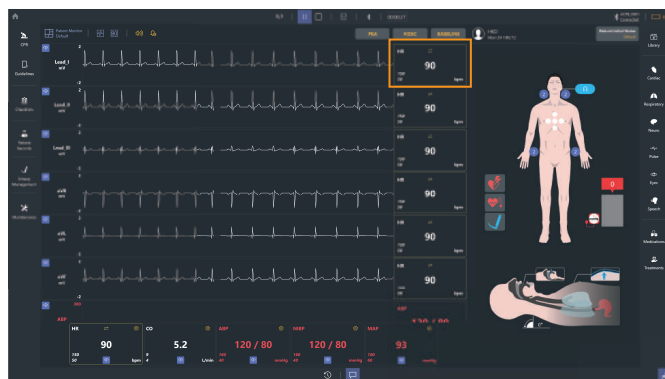
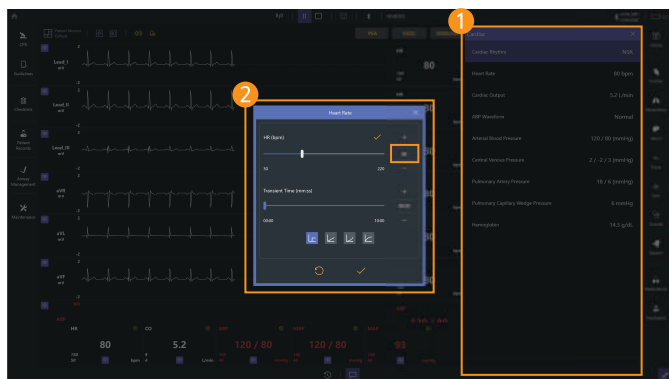


- 1 Select patient parameter menu
 - Select the patient parameter menu that you want to change.
- 2 Set the parameter
 - Change the parameter and transient time.
 - When the change is made on patient parameters,  will be displayed.
- 3 Reset the parameter
 - The patient parameters can be reset using the reset icon.

7-11. Scenario mode

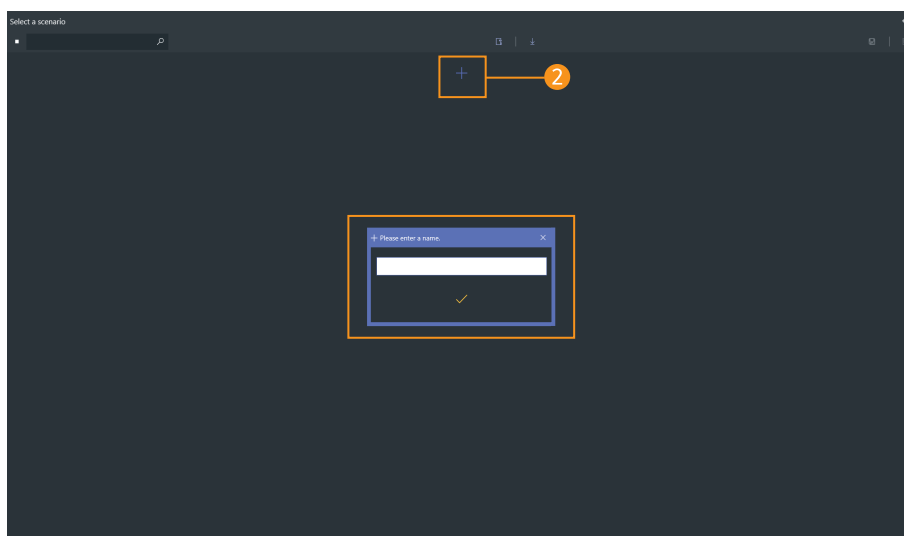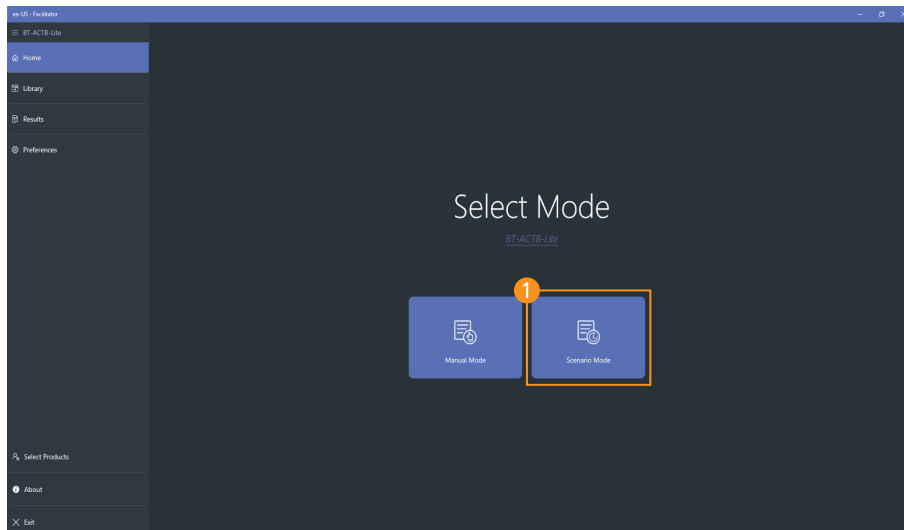

- 1 Start scenario mode
 - Select the scenario mode from the home screen.
- 2 Create a scenario
 - Click the **Add scenario button** and enter a scenario name.


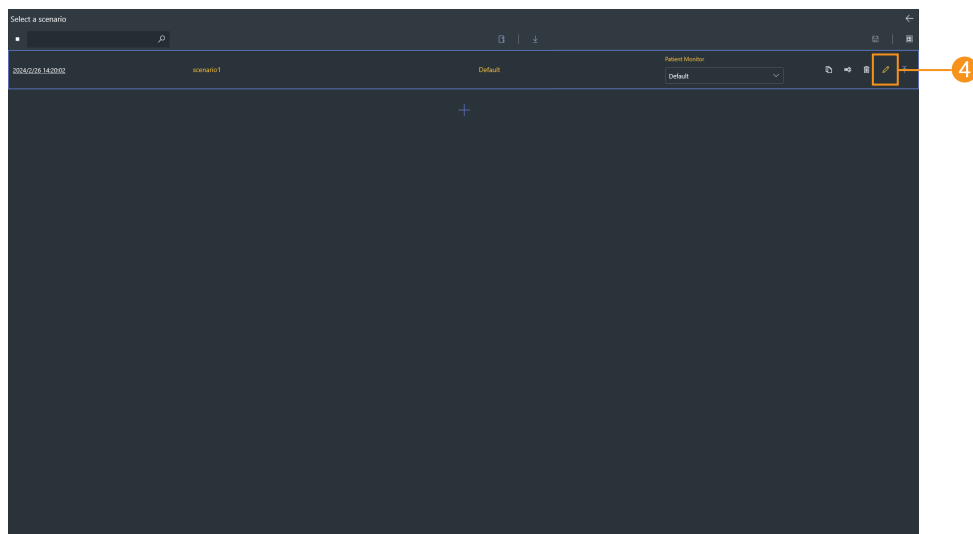
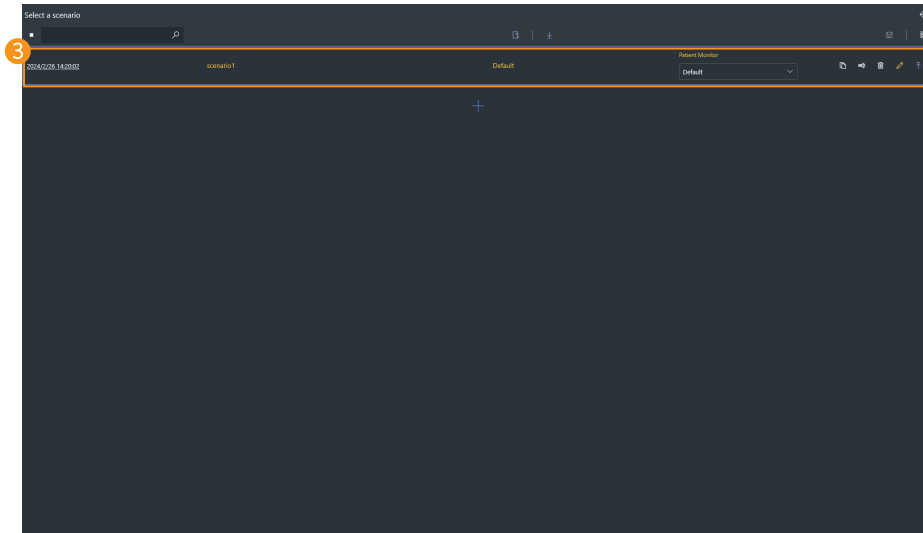


3 Confirm scenario creation

- You can check the scenario created date and name.
- You can check the selected initial patient status.
- You can change the scenario name.
- You can delete the scenario.
- You can copy the scenario.
- You can export or import scenarios.

4 Edit the scenario

- Click the **Edit button** and edit the scenario.

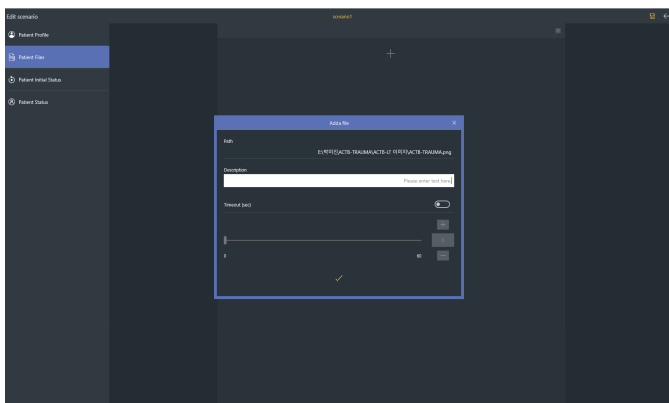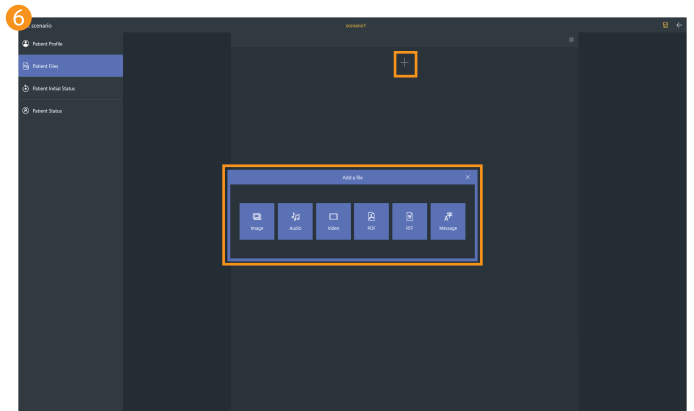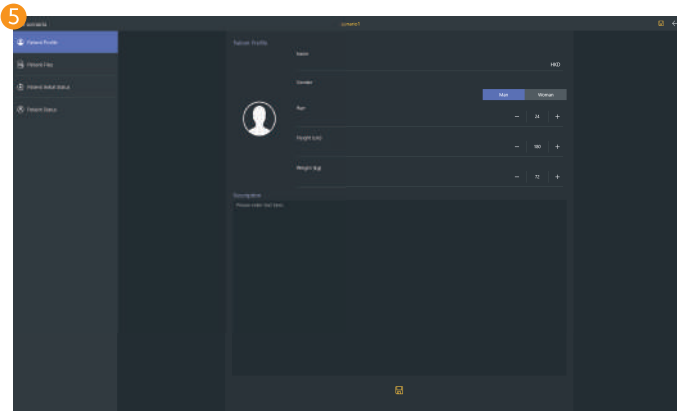


5 Set the patient profile




- Set the patient's name, gender, age, height and weight.
- A brief record of the patient's condition is possible.

6 Add patient file





- File can be saved as image, audio, video, PDF, RTF, text format.
- When adding the file, you can add the brief description and set the display expiration time.

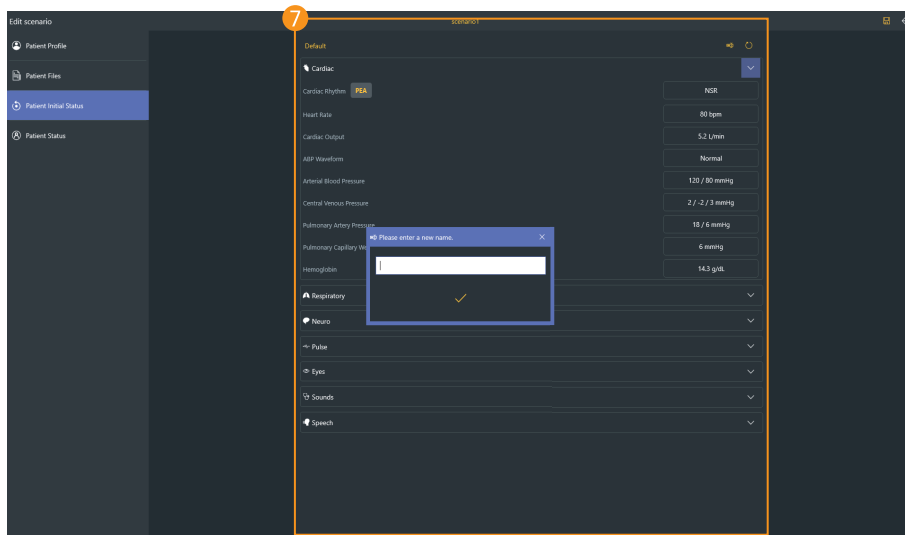

- 7 Set the initial patient status
 - You can set or save the initial patient status.
 - Set the name of the initial patient status.

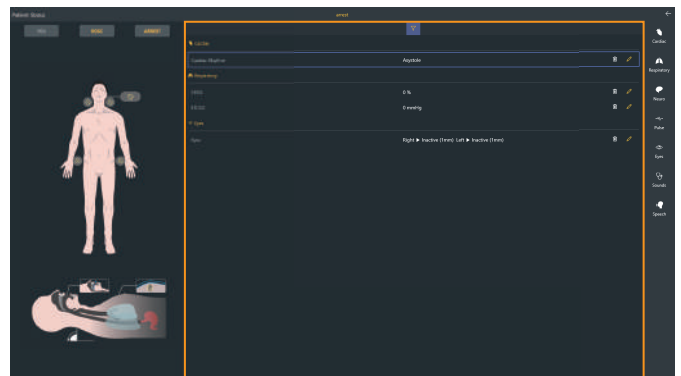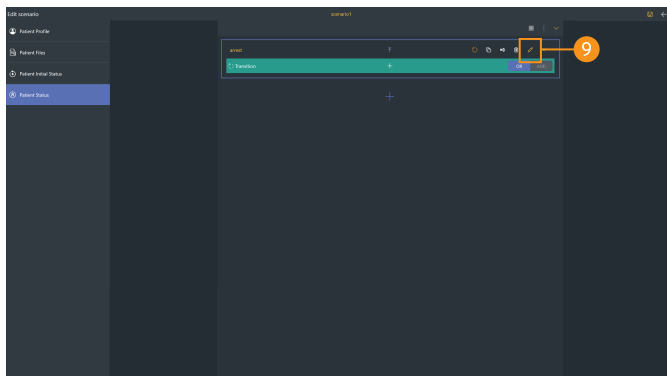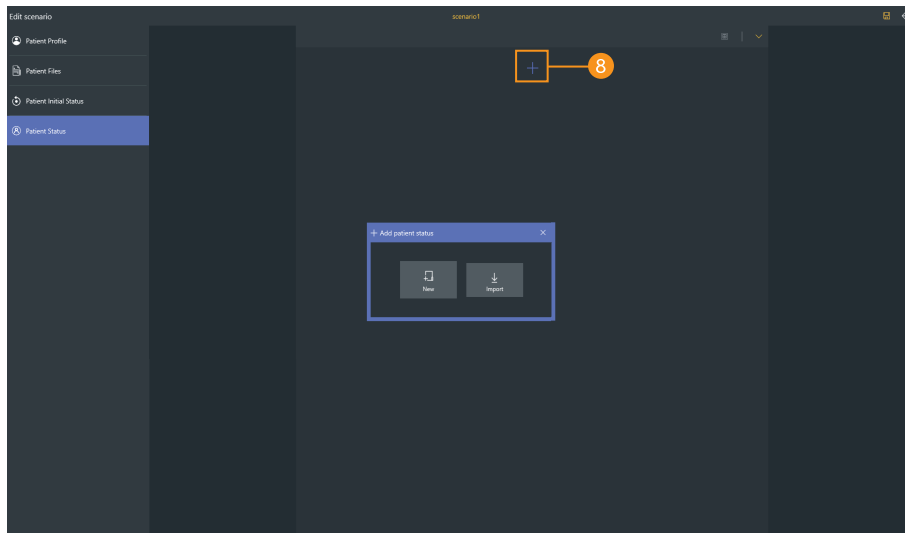

- 8 Add patient status
 - Click the **Add button** to create a patient status needed for the training.
 - You can get patient status from the patient status library.
- 9 Set the patient status
 - Click the **Edit button** to change the patient status.

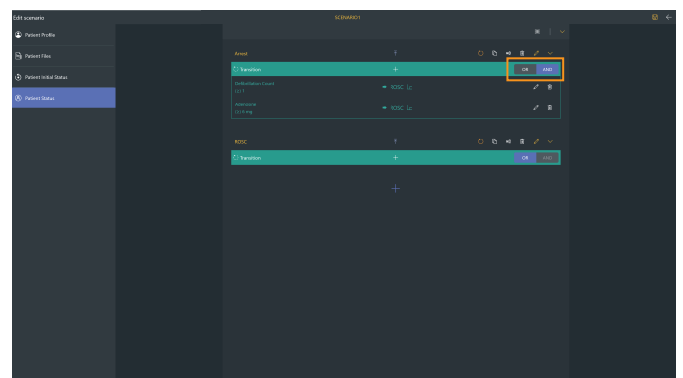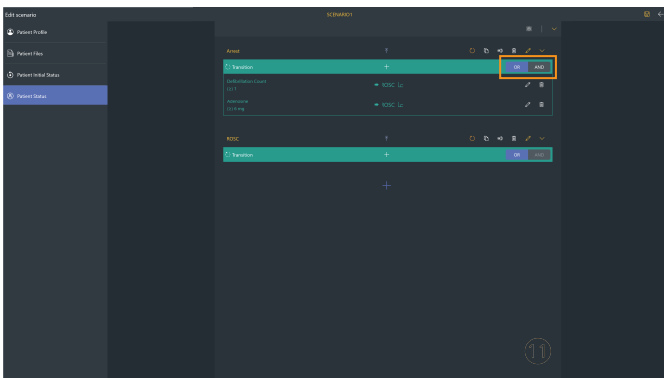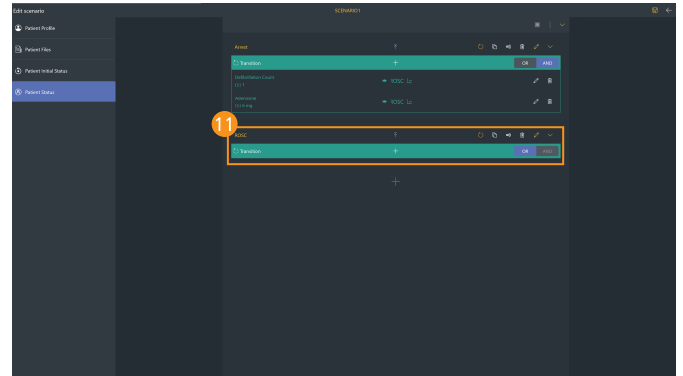
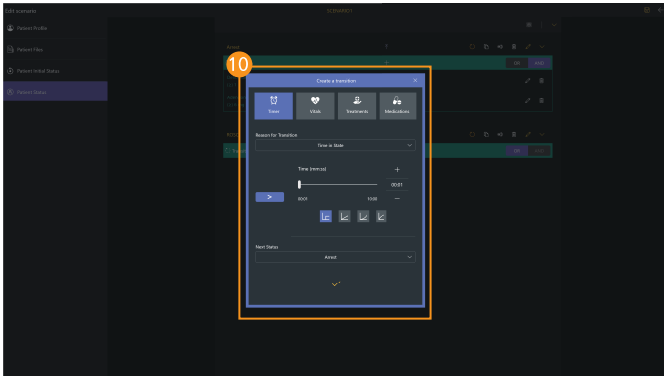


10 Create transition

- You can create transition conditions for changing patient status.
- There are timer, vital signs, treatments and medications for the transition.
- OR and AND function are possible between added transitions.
 - OR : The transition will be proceeded even if only one transition condition is satisfied.
 - AND : The transition will be proceeded when all transition conditions are satisfied.

11 Check the transition

- Check the transition condition and the next patient status.

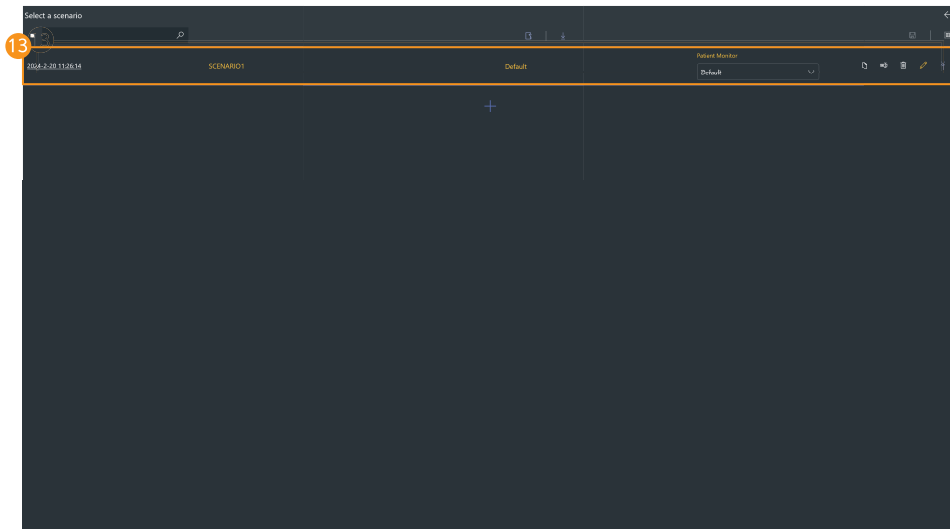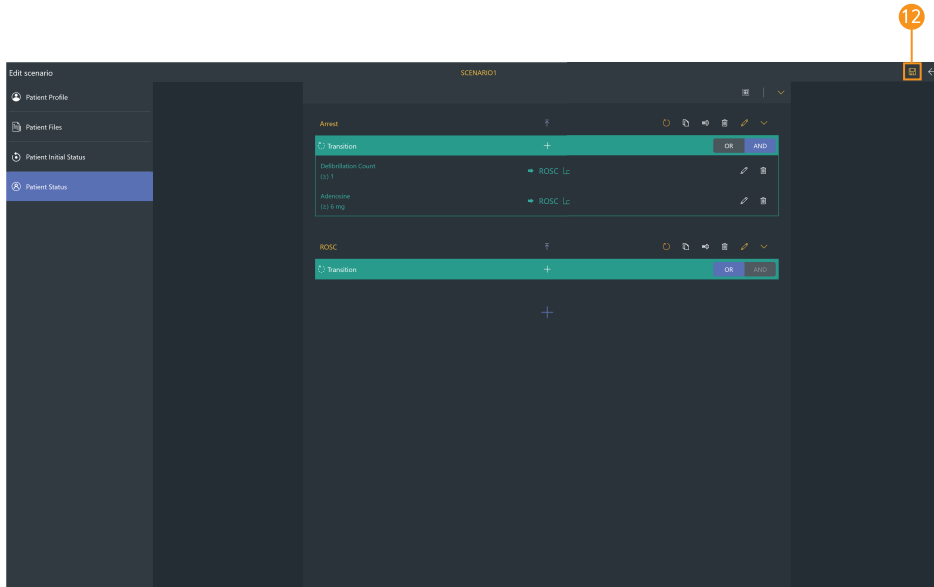


12 Save the scenario

– Click the **Save button** to save the scenario.



13 Start scenario

– Select the scenario that you want to run from the scenario list.

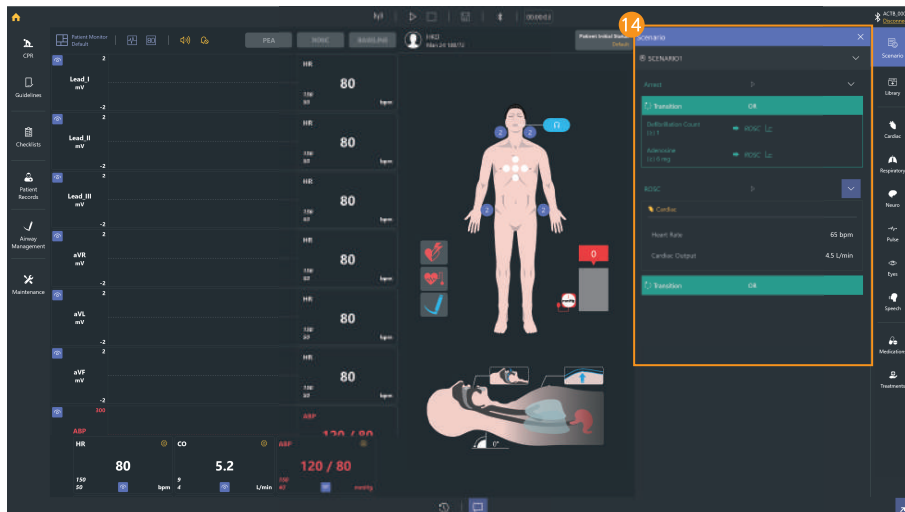

14 Run scenario – 1





- You can start or stop the scenario from the scenario menu.
- Check the summarized information for the patient parameters.

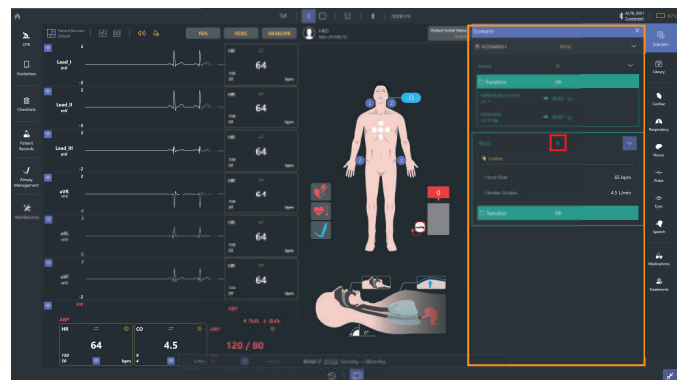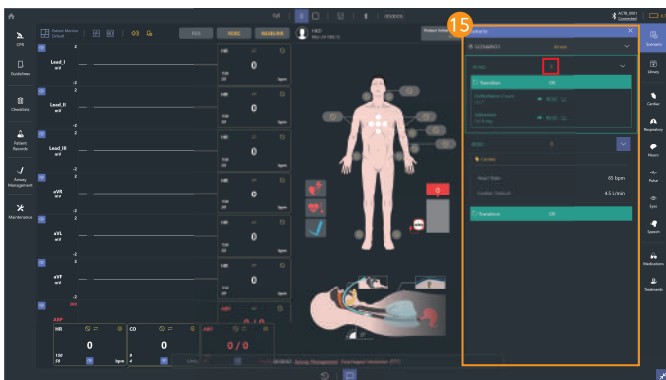

15 Run scenario – 2



- You can check the set patient status.

7-12. Library

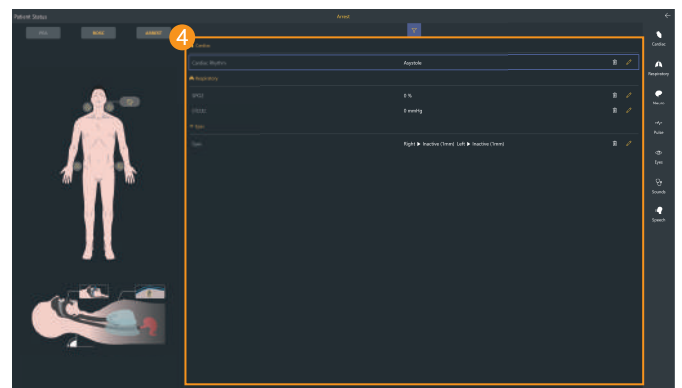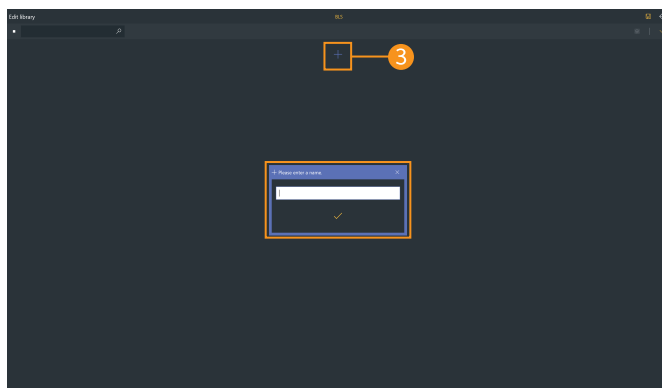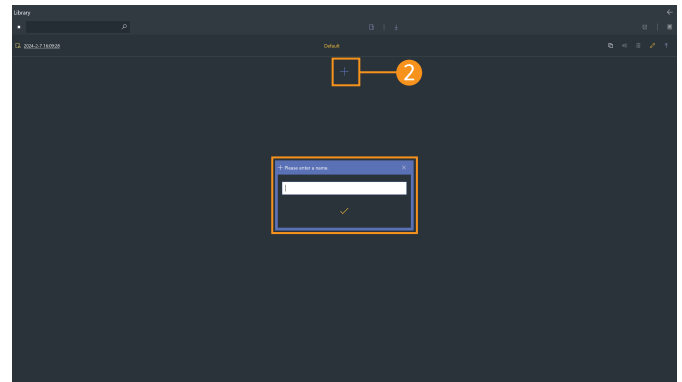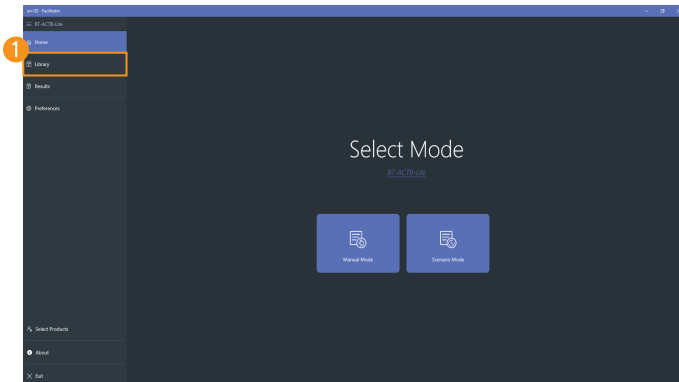

- 1 Select the library menu
 - Select Library from the home screen.
- 2 Create the group for the library
 - Click the **Add group button** and enter a library name.
- 3 Create a library
 - Click the **Add library button** and enter a library name.
- 4 Edit the library
 - You can change or save the patient status by clicking the **Edit button**.
 - You can search for saved patient status.
 - You can change the saved patient status name.
 - You can delete saved patient status.
 - You can copy saved patient status.
 - You can export or import saved patient status.

7-13. View results

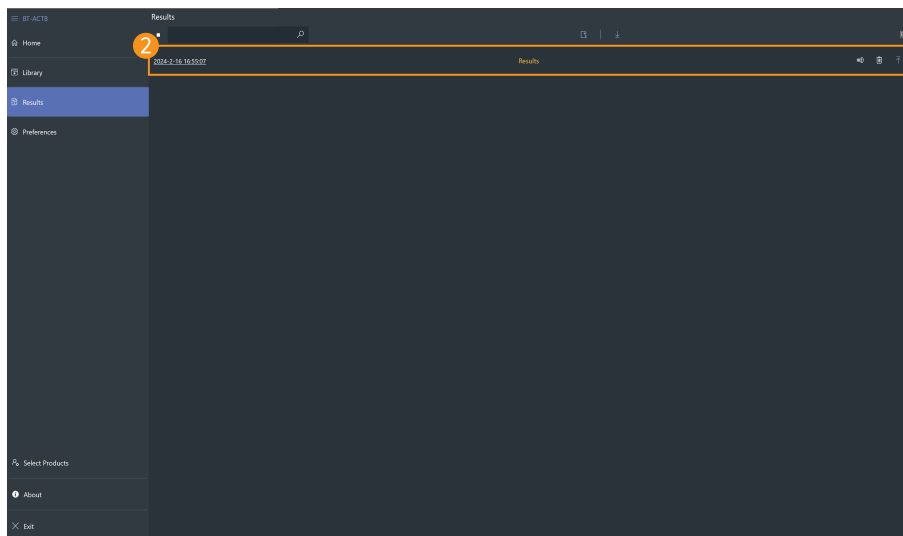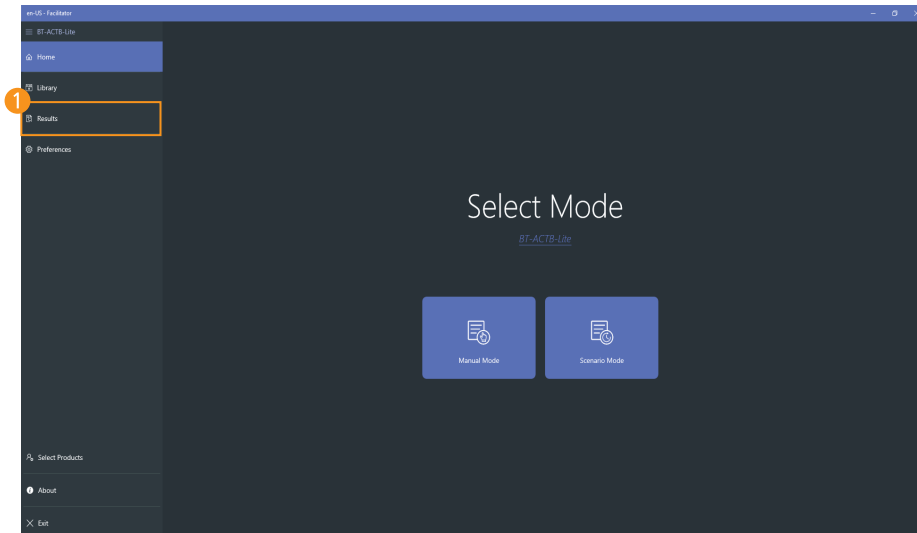

- 1 Select result viewer
 - Select result viewer from the home screen.
- 2 View saved results
 - Select the desired result from the result list.
 - You can search saved results.
 - You can change the saved result name.
 - You can delete saved results.
 - You can export or import saved results.

Event Log	Event Log	CPR Graph	CPR Chart
000000	Patient Status	Patient Status	Patient Initial Status
000002	Simulator	Airway Management	Jaw Thrust
000002	Simulator	Airway Management	Esophageal Intubation (ETI)
000012	Parameter	Heart Rate	80bpm - 50bpm (Sec)
000017	Parameter	Heart Rate	50bpm
000021	Parameter	Arterial Blood Pressure	120/80 (mmHg) - 91/42 (mmHg)
000024	Parameter	Cardiac Rhythm	NSR - VF (Mcp)
000029	Treatments	Defibrillator	Biphasic Defibrillation ▶ 150 (J)
000035	Parameter	Cardiac Rhythm	VF (Mcp) - NSR
000043	Parameter	Heart Rate	80bpm - 90bpm (Sec)
000048	Parameter	Heart Rate	90bpm
000051	Treatments	Fraction of Inspired Oxygen	FiO2 ▶ 10%
000054	Treatments	Need Canula	Flow Rate ▶ 2LPM
000056	Parameter	Blood Temperature	37°C - 34.3°C




Time	Compression Depth (mm)	Hands Placement (C/A/D/L/R)	Compression Rate (bpm)	Complete Recoil (True/False)
Time	Ventilation Volume (ml)	Inspiratory Time (sec)	Ventilation Interval (sec)	Stomach Inflation (True/False)
00:00:44.55	70	C	0	True
00:00:44.95	70	C	109	False
00:00:45.40	70	C	171	False
00:00:45.70	64	C	150	False
00:00:46.00	68	C	200	False
00:00:46.35	63	C	171	False
00:00:46.65	63	C	200	False
00:00:46.95	56	C	171	False
00:00:47.20	43	C	240	False
00:01:00.75	570	0.4	0	False
00:01:00.75	300	0.4	1.4	False
00:01:10.25	70	C	0	True
00:01:10.60	70	C	150	False
00:01:10.95	70	C	200	False
00:01:11.20	52	C	200	False
00:01:11.50	68	C	240	False
00:01:11.80	56	C	200	False
00:01:12.20	67	C	171	False
00:01:12.55	67	C	46	True
00:01:14.05	68	C	130	False
00:01:14.50	70	C	120	True
00:01:14.85	60	C	150	False
00:01:17.45	250	0.3	0.3	False
00:01:19.05	480	0.8	1.5	False

3 Review the event logs


- Review the data such as changed parameters during the training.
- You can view event logs by grouping them.

4 Export to Excel file

- You can export the event logs as an Excel file format.
- You can edit the event logs in Excel or save it with a another name.

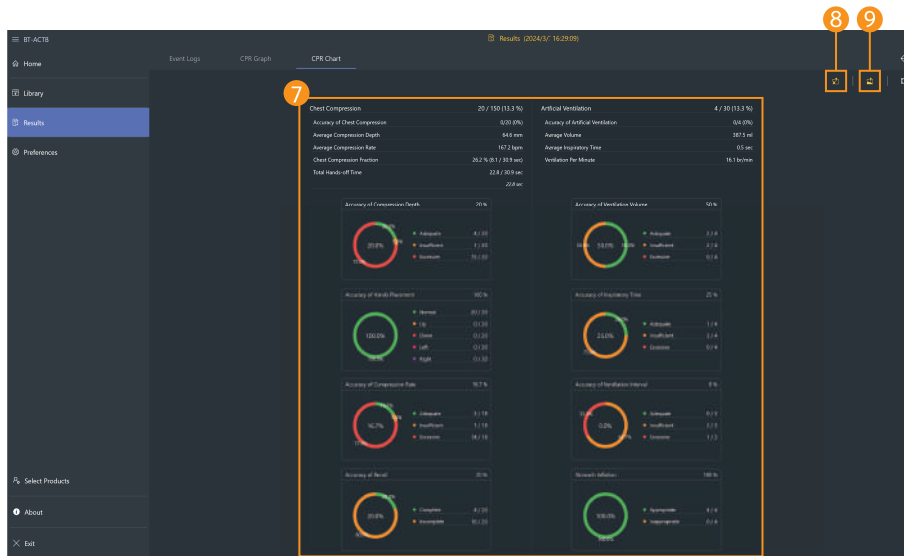

Time	Compression Depth (mm)	Hands Placement (C/A/D/L/R)	Compression Rate (bpm)	Complete Ratio (True/False)
	Ventilation Volume (ml)	Respiratory Time (sec)	Ventilation Interval (sec)	Stomach Inflation (True/False)
00:00:01.15	76	C	97	False
00:00:01.40	70	C	177	False
00:00:01.70	66	C	150	False
00:00:02.00	69	C	200	False
00:00:02.30	63	C	177	False
00:00:02.60	60	C	200	False
00:00:02.90	56	C	177	False
00:00:03.20	43	C	240	False
00:00:03.50	340	R/L	0	True
00:00:03.75	600	R/L	1.4	True
00:00:04.05	70	C	0	True
00:00:04.30	70	C	150	False
00:00:04.60	70	C	200	False
00:00:04.90	54	C	200	False
00:00:05.20	66	C	240	False
00:00:05.50	56	C	200	False
00:00:05.80	67	C	177	False
00:00:06.10	67	C	46	True
00:00:06.40	66	C	100	False
00:00:06.70	70	C	100	True
00:00:07.00	60	C	150	False
00:00:07.30	250	R/L	0.3	False
00:00:07.60	400	R/L	1.5	False

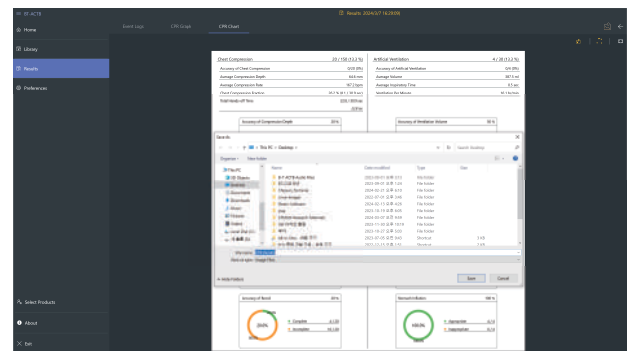

7 View CPR chart

- You can see the CPR results as a chart.

8 Export to Excel file

- You can export CPR data in Excel file format.

9 Export to image file

- You can export or print CPR results in image file format.


8. Monitor Feature






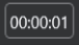
8-1. Patient monitor

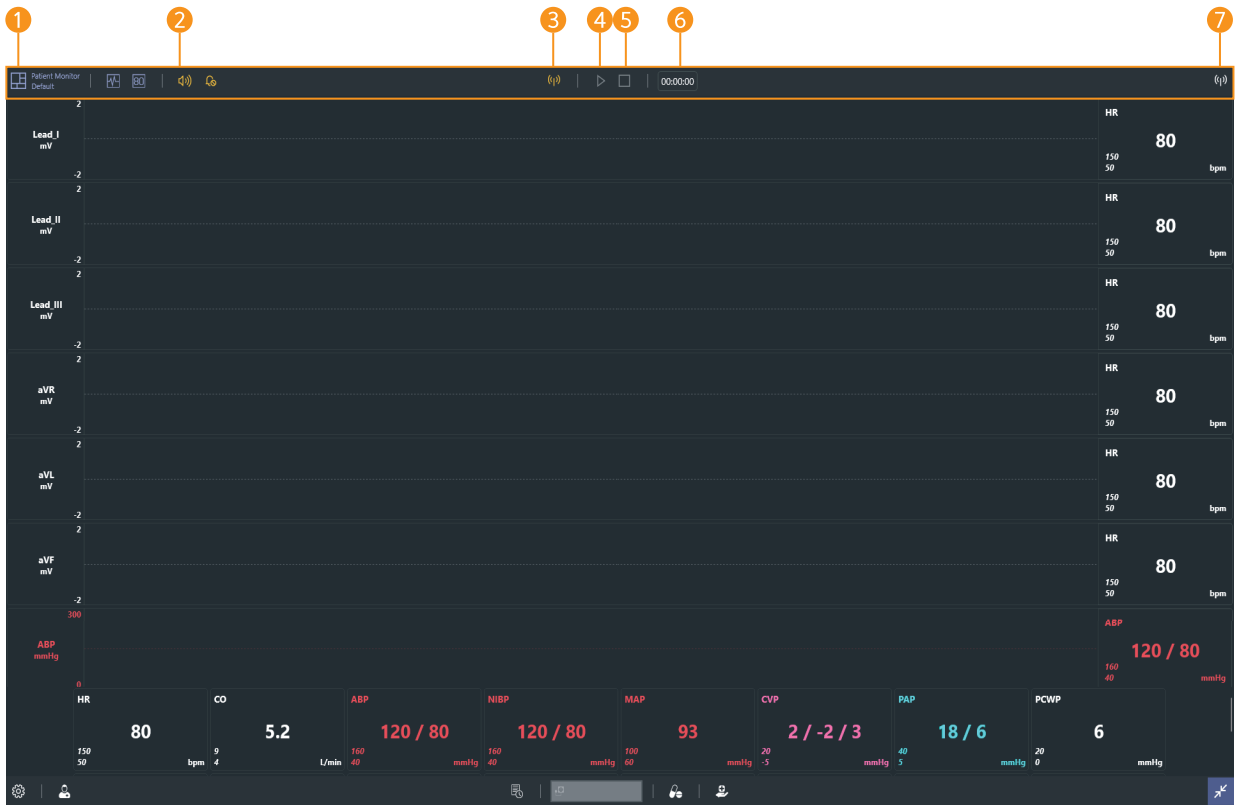


1 Top menu 2 Bottom menu 3 Patient monitor







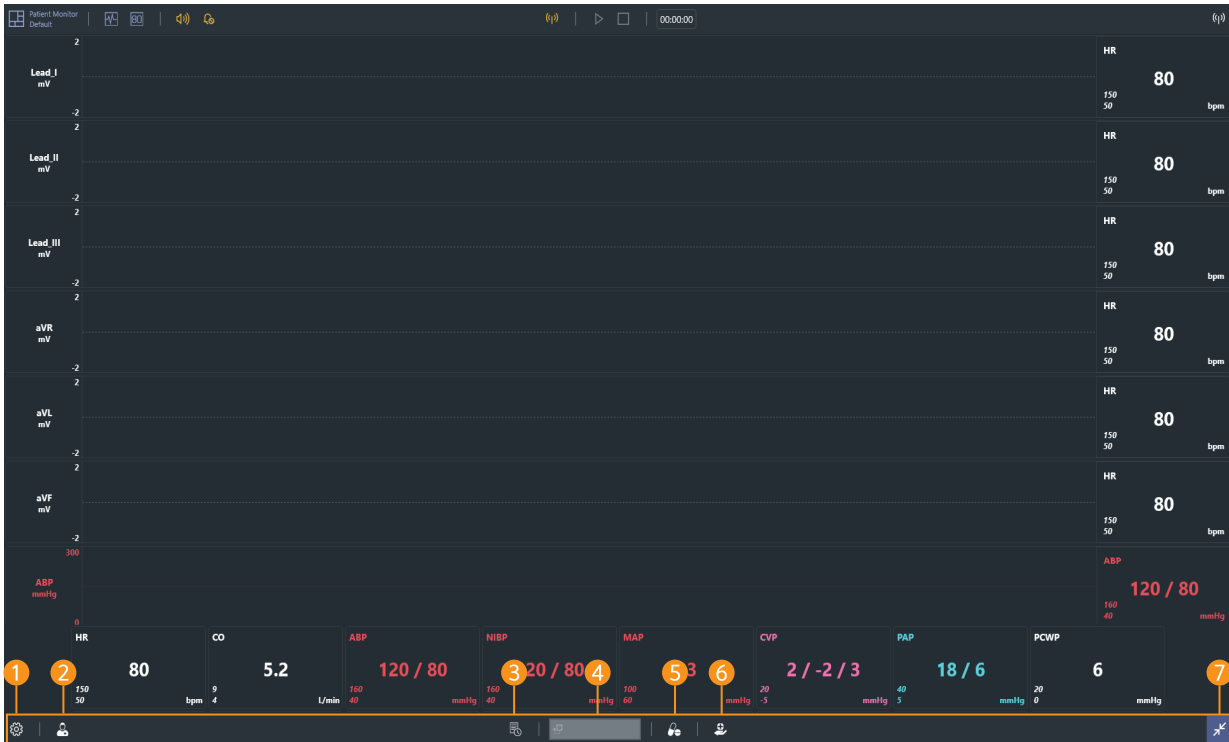

8-2. Top menu







- ① **Patient monitor settings**  : Change settings of the layout of patient monitor, the audio and NIBP.
- ② **Mute**  : Turn the sound ON/OFF.
- ③ **Wi-Fi settings**  : Connect with Facilitator.
- ④ **Start training**  : Displays the start and stop status of the training.
- ⑤ **End training**  : Displays the status of the end of training.
- ⑥ **Training time**  : Display the training time.
- ⑦ **Wi-Fi connection status** : Display the connection status with the Facilitator.

8-3. Bottom menu




- 1 **Settings**  : You can check the preferences and S/W information.
- 2 **Patient records**  : You can check the patient profile, patient history and diagnostic result.
- 3 **Patient status records**  : You can record or print the current patient status.
- 4 **NIBP**  : You can start measuring NIBP.
- 5 **Medications**  : You can choose and administer the medications needed for the training.
- 6 **Treatments**  : You can train defibrillation, pacing, oxygen therapy.
- 7 **Full-screen mode**   : You can switch to full screen mode.

8-4. Patient monitor settings

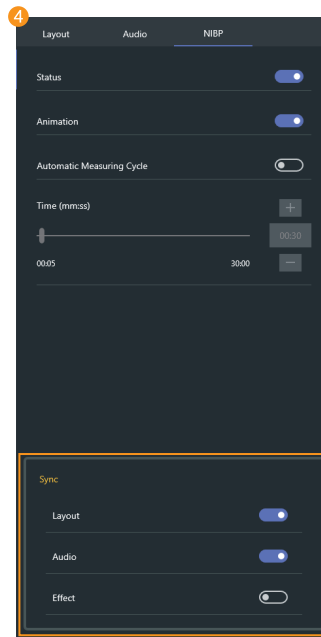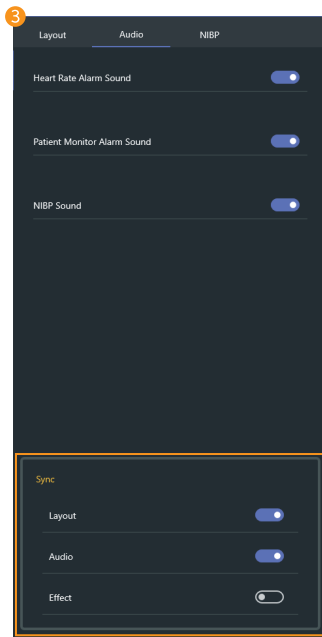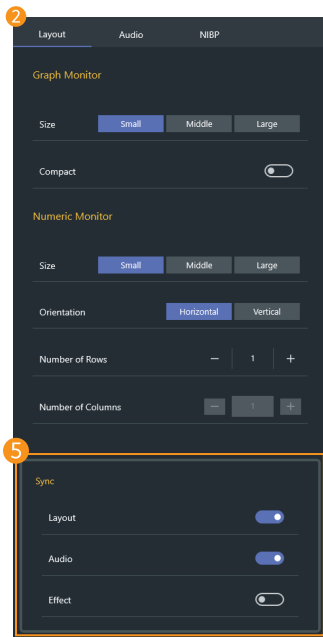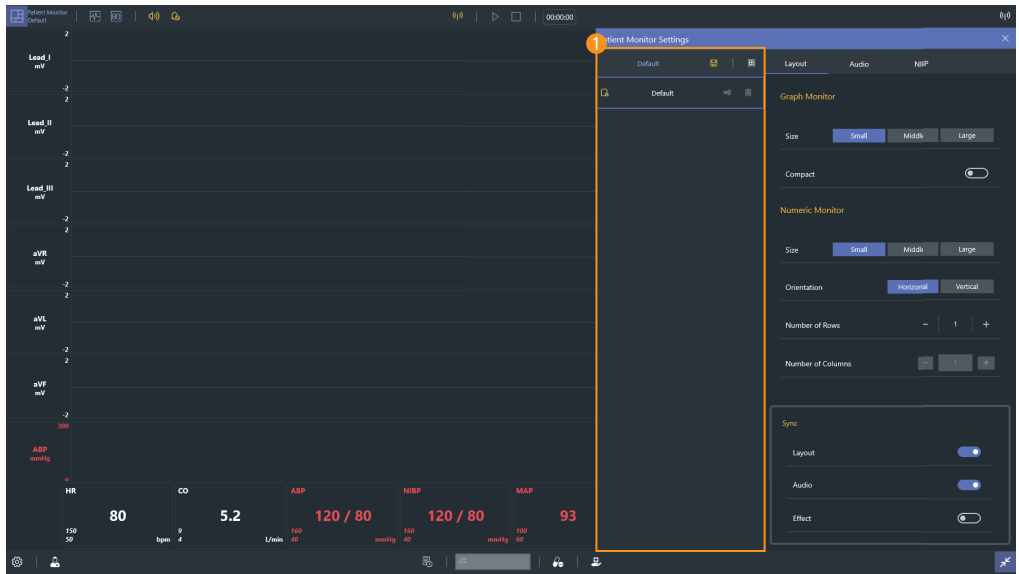

- 1 Select the patient monitor
 - Select or create a saved patient monitor.
- 2 Patient monitor layout settings
 - You can set the layout of the graph monitor and numeric monitor.
- 3 Audio settings
 - You can set the heart rate alarm, patient monitor alarm, and NIBP sound.
- 4 NIBP settings
 - You can set the NIBP auto cycling.
- 5 Sync setting
 - You can select the Sync setting of each.

8-5. Patient monitor (Graph monitor)

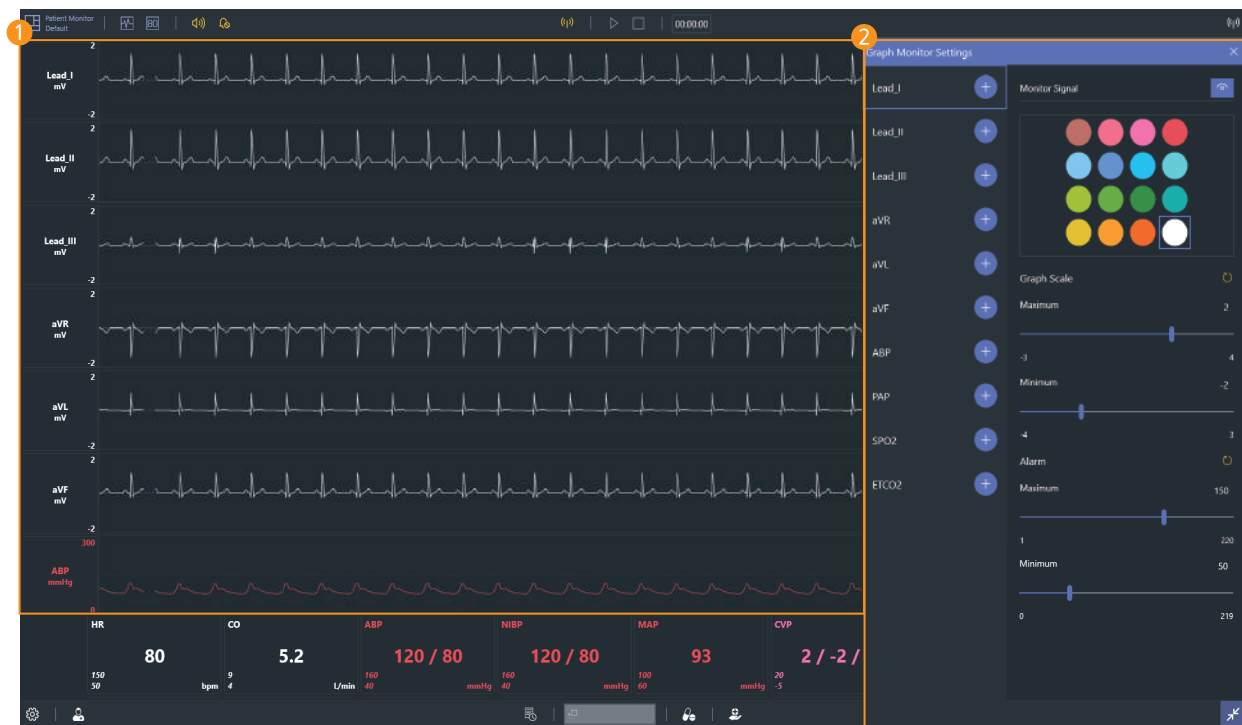


1 Graph monitor

- Display ECG, ABP, PAP, SPO2, ETCO2 waveform.

2 Graph monitor settings

- You can add or delete waveforms to the graph monitor.
- You can change the color of the waveform.
- You can display or block the signal of the waveform.
- You can adjust the graph scale.
- You can set the alarm.

8-6. Patient monitor (Numeric monitor)

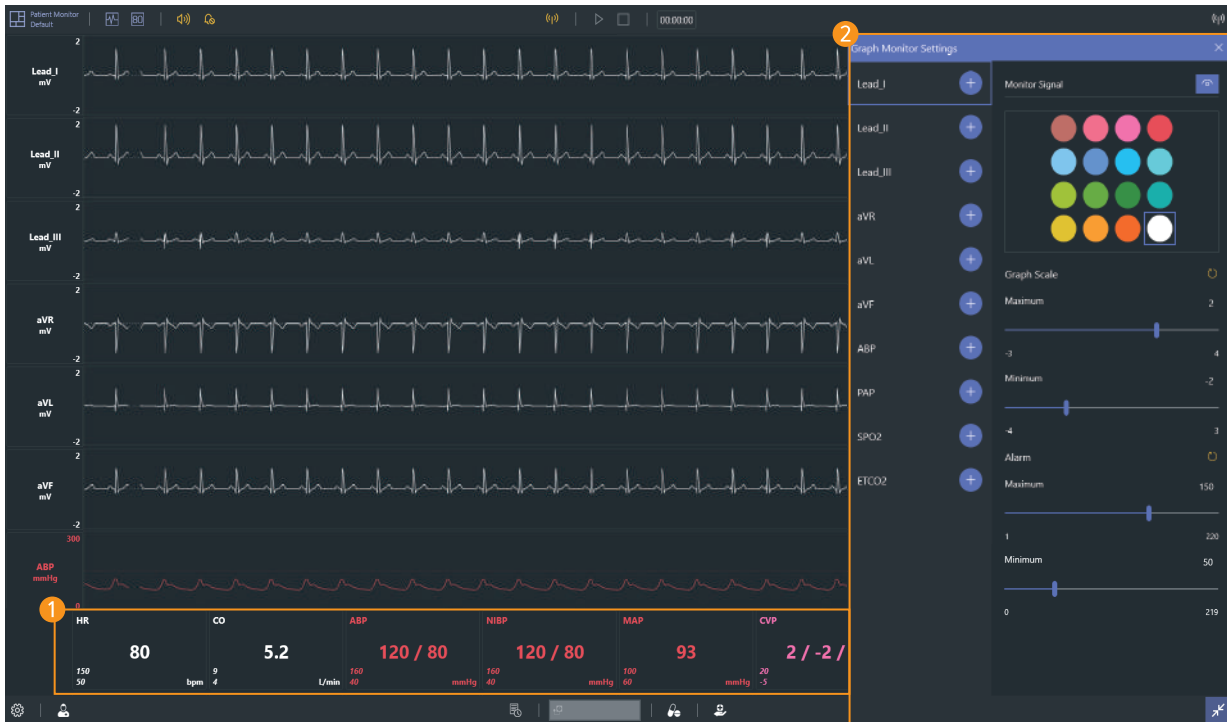


① Numeric monitor

- Display numeric values such as HR, CO, ABP.

② Numeric monitor settings

- You can add or delete patient parameters in the numeric monitor.
- You can change the color of the patient parameters.
- You can show or hide patient parameters.
- You can set the alarm.

8-7. Patient records

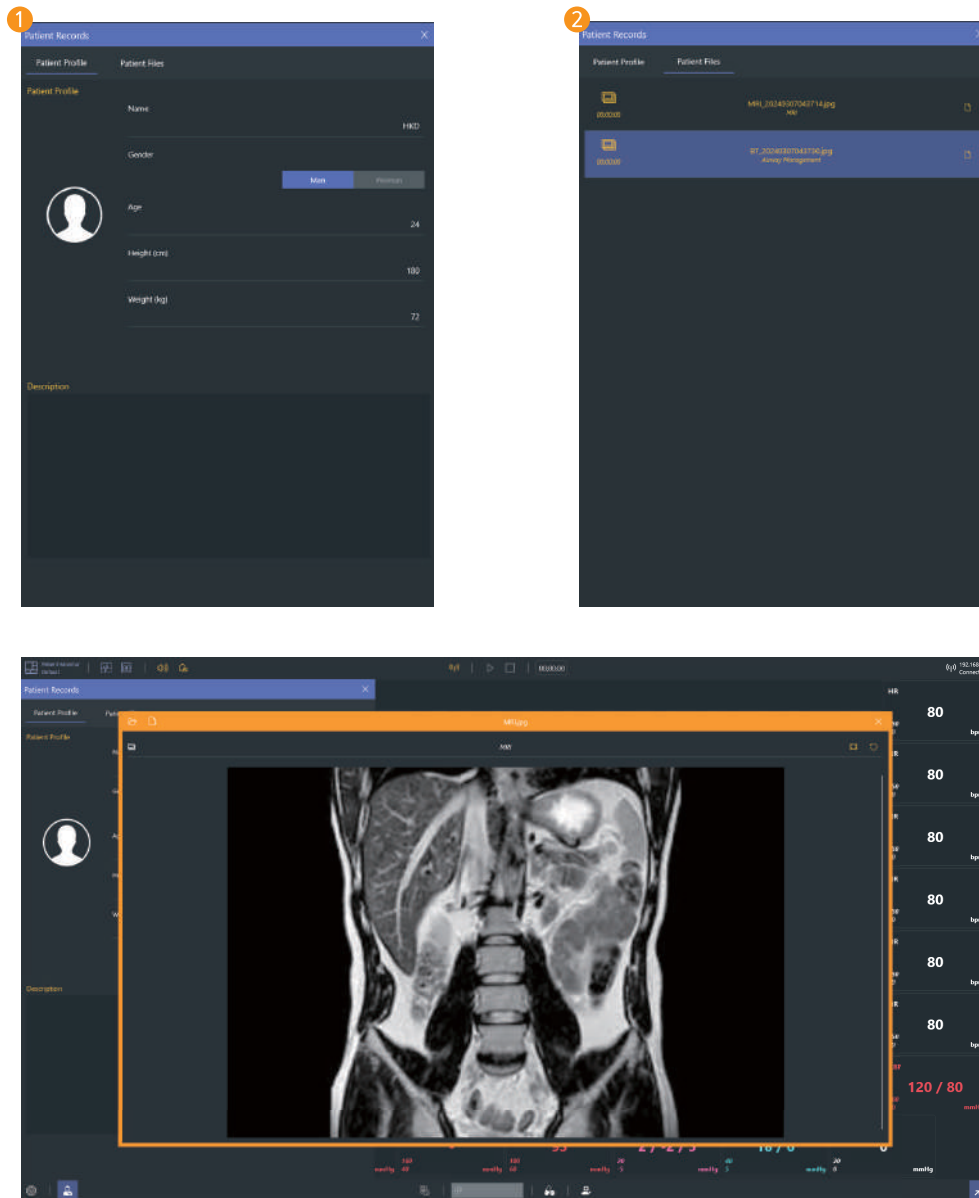

1 Patient profile


– You can check the current patient profile.

2 Patient files

– Patient files sent from Facilitator are saved.

– Click the **Open file button** to view the patient file.



8-8. NIBP

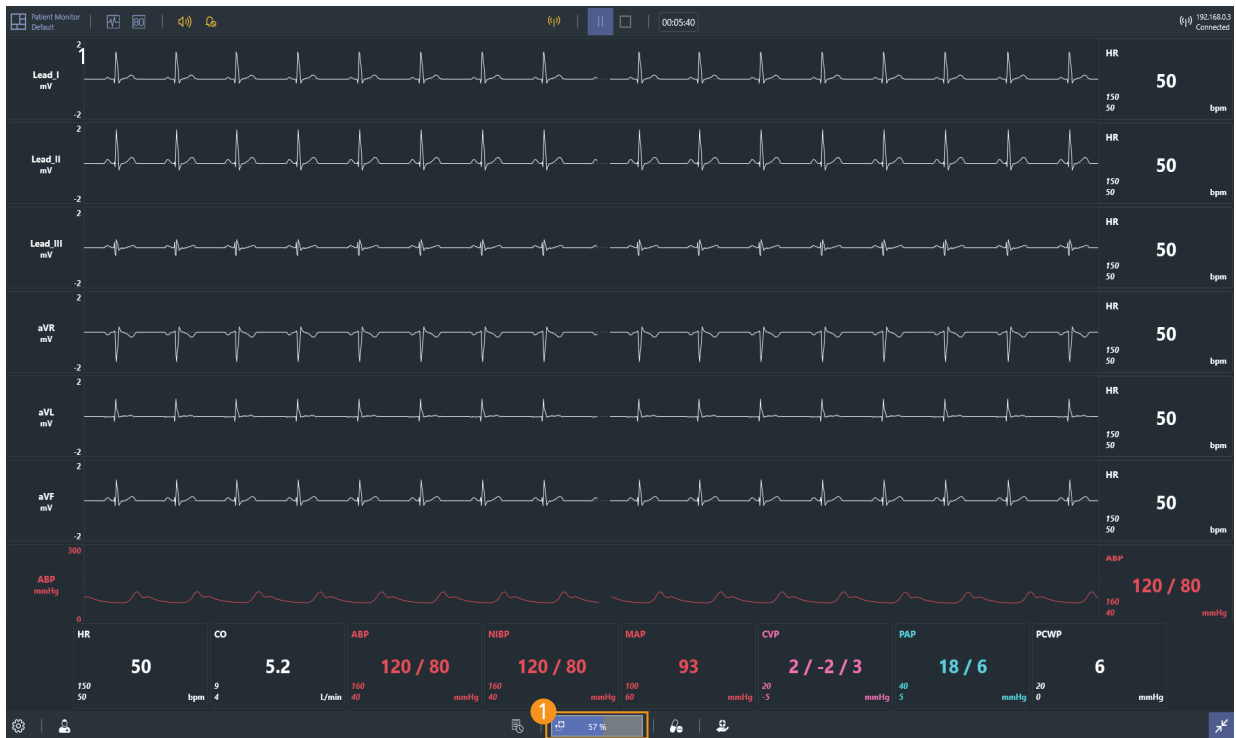

1 Measure NIBP


- Click the mouse to start the NIBP measurement.
- NIBP is automatically measured when the NIBP auto cycling is set.

8-9. Medications

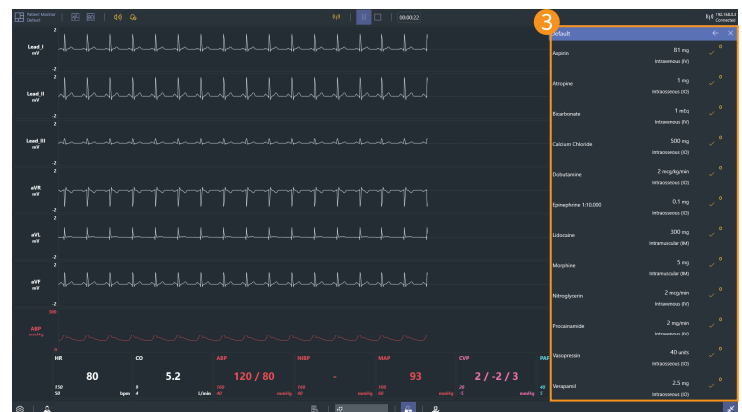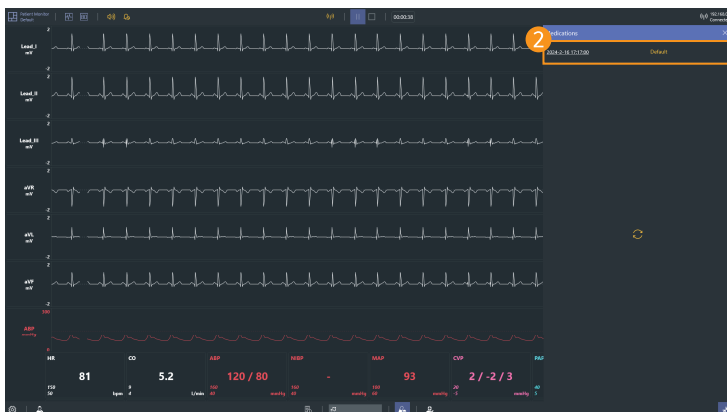

- 1 Bring medications
 - Click the **Import medication button** to import medication stored in the Facilitator.
- 2 Select medication
 - You can choose the medication you need for your training from the medication list.
- 3 Medication administration
 - You can choose the type of medication and set the dosage and medication method.

8-10. Defibrillator

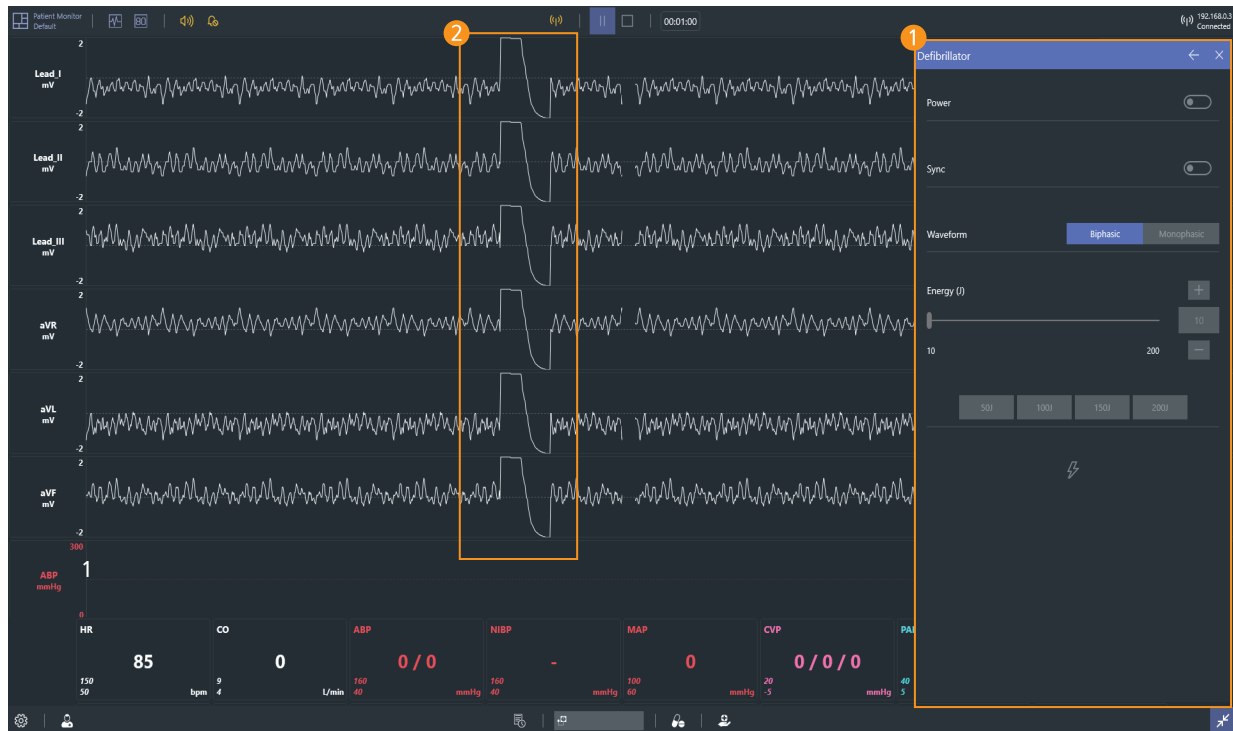


1 Defibrillator

- You can train synchronous and asynchronous defibrillation.
- You can select the type and energy of the defibrillation waveform.

2 Defibrillation waveform

- Defibrillation waveforms are displayed on the ECG graph during defibrillation.

8-11. Pacer

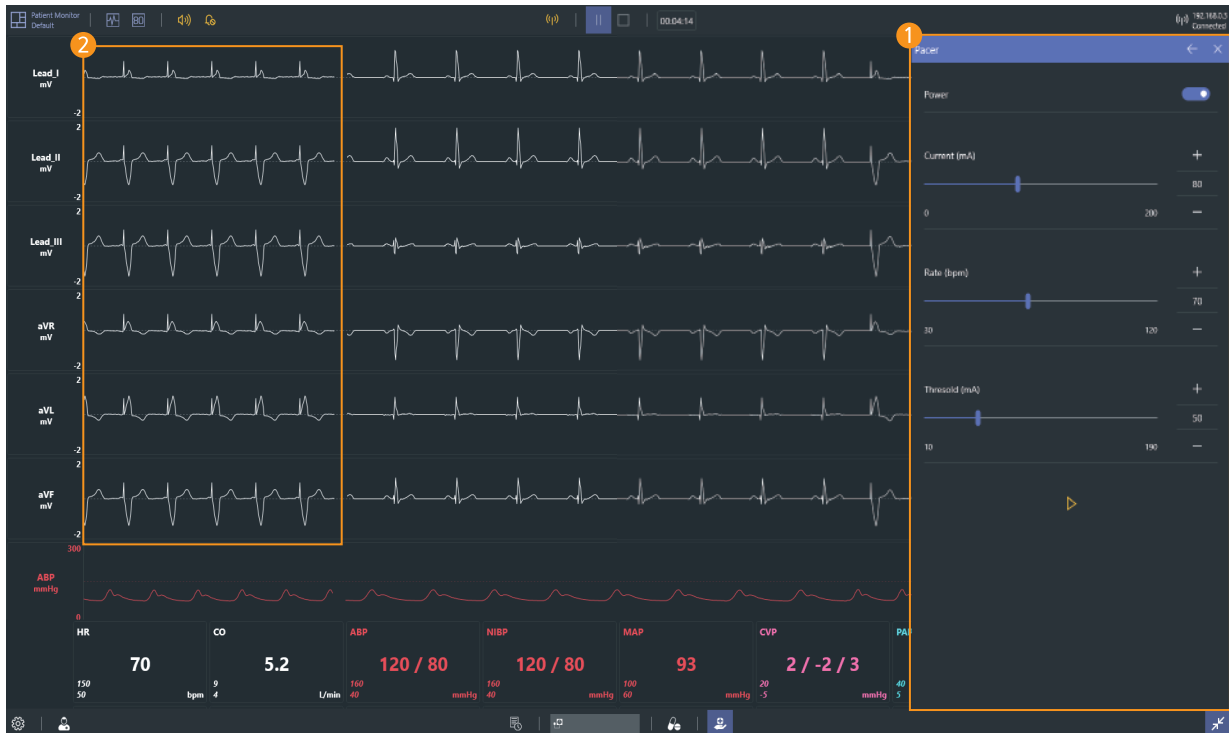


1 Pacer

- You can set pacing current and rate.
- You can set the patient threshold current.

2 Capture beats



- When pacing current is applied, capture beats are displayed on ECG graph.

8-12. Oxygen therapy

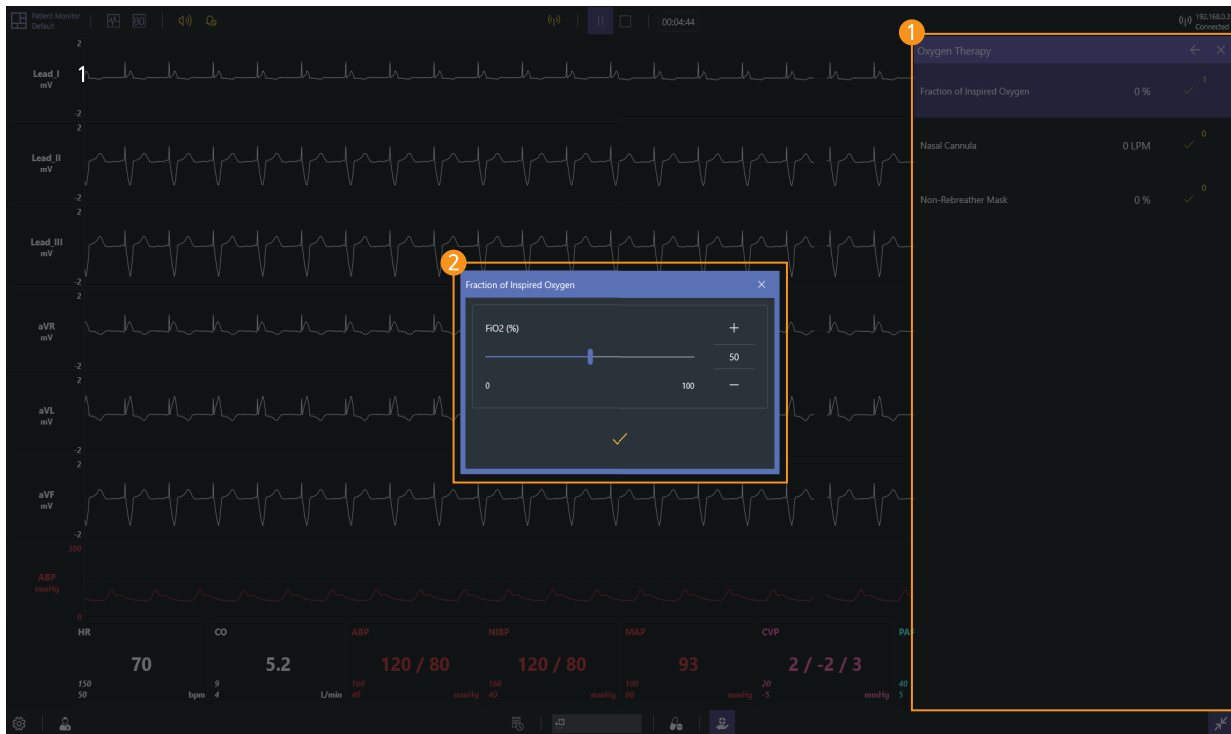

- 1 Select oxygen therapy
 - You can choose the oxygen therapy that such as fraction of inspired oxygen and nasal cannula.
- 2 Set the oxygen therapy
 - You can set appropriate FiO2 and flow rate for the patient treatment.

8-13. Patient status records

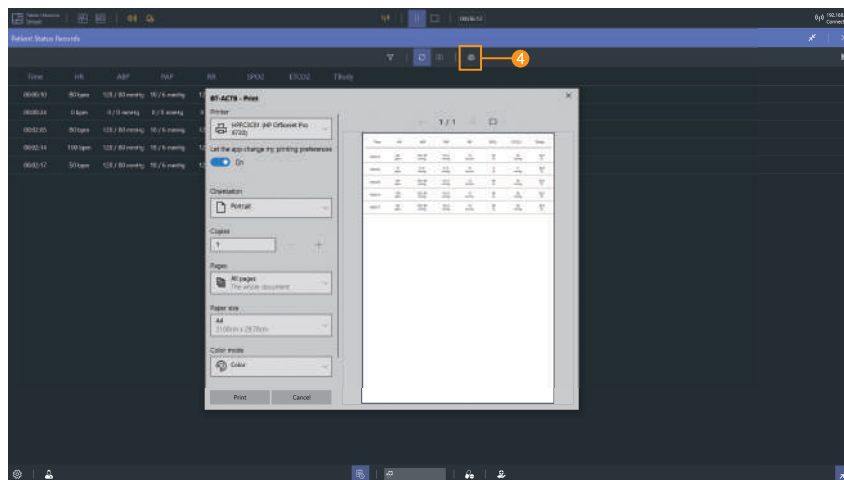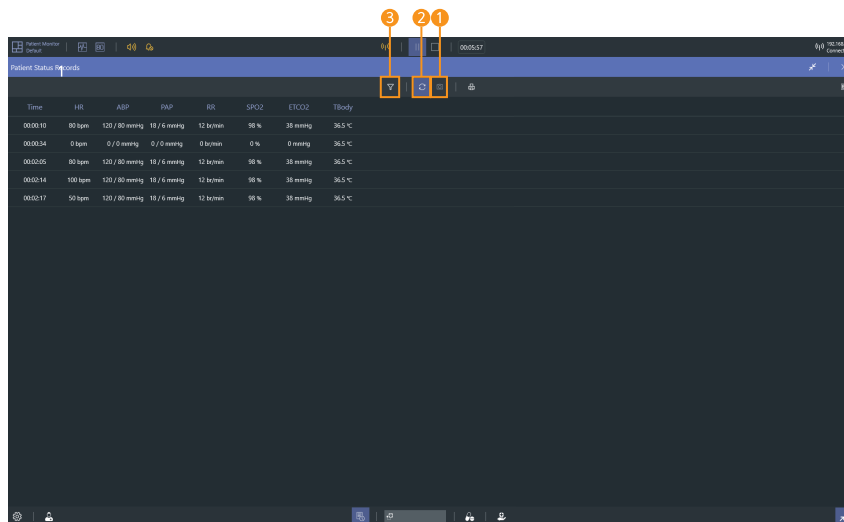

- 1 Patient status records
 - Click the **Capture button** to record the current patient status.
- 2 Auto record
 - Automatically recorded when patient status changes.
- 3 Set filters
 - Use the filter to add or delete the currently recorded patient status parameters.
- 4 Printing the patient status records
 - You can print the patient status record by clicking the **Print button**.

BT-ACTB-L

ALS Training Simulator : Lite

A-313, Samsung Techno-valley, 140, Tongil-ro,
Deogyang-gu, Goyang-si, Gyeonggi-do, 10594, Republic of Korea
Homepage: www.btinc.co.kr / E-mail: bt@btinc.co.kr
Tel: +82-2-2038-2903 ,+82-51-903-1144 / Fax: +82 303-3130-0607

Copyright(c) BT Inc. All rights reserved.