


BT-CSIE-T

# Airway Management Training Model

## USER'S GUIDE

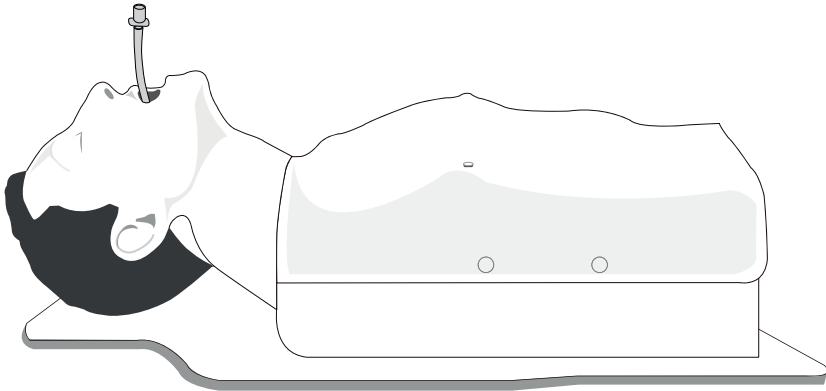

## **Product Information**

Product Name	Airway Management Simulator
Model Name	BT-CSIE-T
Manufacturer	BT Inc.
Country of Origin	Republic of Korea

## **Product Specification**

Dimensions	700 X 300 X 240mm
Weight	8.5kg

## **Customer Service**


Website	<a href="http://www.btinc.co.kr">www.btinc.co.kr</a>
Address	A-313, Samsung Techno-valley, 140, Tongil-ro, Deogyang-Gu, Goyang-Si, Gyeonggi-do, 10594, Republic of Korea
Telephone	+82-2-2038-2903
E-mail	<a href="mailto:bt@btinc.co.kr">bt@btinc.co.kr</a>

# **Cautions**


Please read it carefully to ensure correct usage and user's safety through understanding of the product.

- Ⓞ Use only components provided or approved by BT Inc.
  
- Ⓞ Do not use the product for other purposes described in the manual.
  
- Ⓞ If you disassemble or modify the product without any permission by BT Inc, you cannot get the warranty. Also, making imitated or altered products for commercial purposes are prohibited by law.
- Ⓞ Please avoid extremely hot or cold place and do not store it in a humid place for a long time since it may cause a deformation or a damage to the product.
  
- Ⓞ Never clean the product with chemical substances such as solvent, benzene or thinner, as these may cause ignition or cause the surface deterioration.
  
- Ⓞ BT Inc. can change or adjust the product specification without any notice in order to improve the product.

## Components

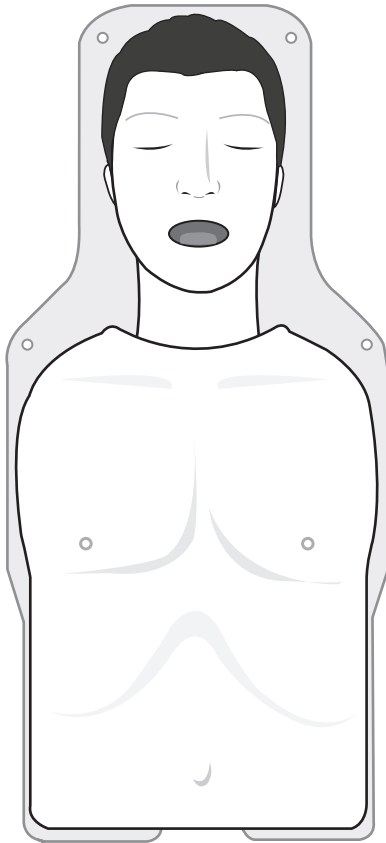

**Adult Torso Manikin**

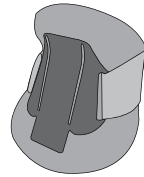

**Cervical collar**



**Lubricant**

\* Etc - Storage case, Skin fixing pin(8ea)


## Consumables & Recommended Clinical Supply Sizes

Consumables	
<ul style="list-style-type: none"> <li>• Skin(face, torso, head)</li> <li>• Lubricant</li> </ul>	<ul style="list-style-type: none"> <li>• Airway for replacement</li> </ul>


Recommended Clinical Supply Sizes	
ETT	I.D. 7.0~8.0mm
I-gel / LMA	#3, #4
Combi-tube	37Fr
OPA	88mm(Green)
NPA	6~7(24~28Fr)

- The following clinical supply sizes are recommended for use with BT-CSIE-T model.
- Other sizes may cause damage to the products.
- Lifetime of the consumables may differ depending on the skill level of the users.


# Basic Functions of Manikin

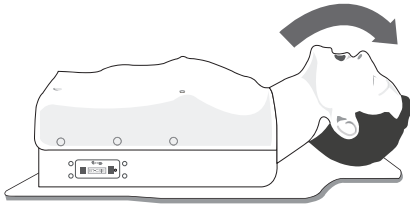


1. Head tilt

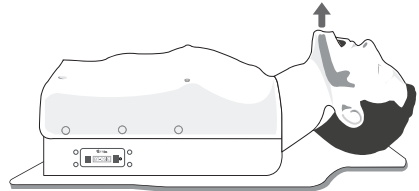


2. Jaw thrust maneuver

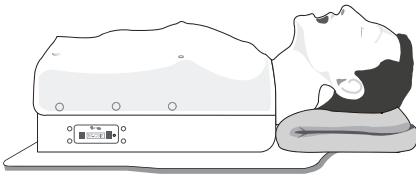

3. Sniffing position

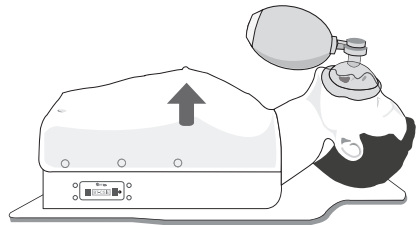


4. Ventilation with the bag valve mask

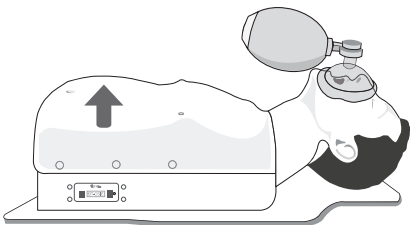

5. Gastric inflation

\* The gastric inflation may happens by too strong, too much, too frequent ventilaion or esophageal intubation.

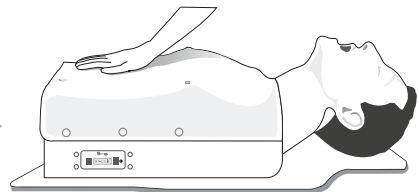

6. Deflation by hand

\* If the gastric inflation happens, press the abdomen softly to discharge the air.

BT-CSIE-T

# Airway Management Training Model

A-313, Samsung Techno-valley, 140, Tongil-ro,  
Deogyang-gu, Goyang-si, Gyeonggi-do, 10594, Republic of Korea

Homepage: [www.btinc.co.kr](http://www.btinc.co.kr) / E-mail: [bt@btinc.co.kr](mailto:bt@btinc.co.kr)  
Tel: +82-2-2038-2903, +82-51-903-1144 / Fax: +82 303-3130-0607

Copyright(c) BT Inc. All rights reserved.