

BT-CSUP-B2

Ultrasound Training Model

USER GUIDE



Product Information

Product Name	Ultrasound Training Model
Model Name	BT-CSUP-B2
Manufacturer	BT Inc.
Country of Origin	Republic of Korea

Product Specification

Dimensions	148 X 165 X 48mm
Weight	0.8kg

Customer Service


Website	www.btinc.co.kr
Address	A-313, Samsung Techno-valley, 140, Tongil-ro, Deogyang-Gu, Goyang-Si, Gyeonggi-do, 10594, Republic of Korea
Telephone	+82-2-2038-2903
E-mail	bt@btinc.co.kr

| Cautions

Please read it carefully to ensure correct usage and user's safety through understanding of the product.

- ① Do not use the product for other purposes described in this manual.
- ② If you disassemble or modify the product without any permission by BT Inc, you cannot get the warranty. Also, making imitated or altered products for commercial purposes are prohibited by law.
- ③ Please avoid extremely hot or cold place and do not store it in a humid place for a long time since it may cause a deformation or a damage to the product.
- ④ Never clean the product with chemical substances such as solvent, benzene or thinner, as these may cause ignition or cause the surface deterioration.
- ⑤ BT Inc. can change or adjust the product specification without any notice in order to improve the product.

Components

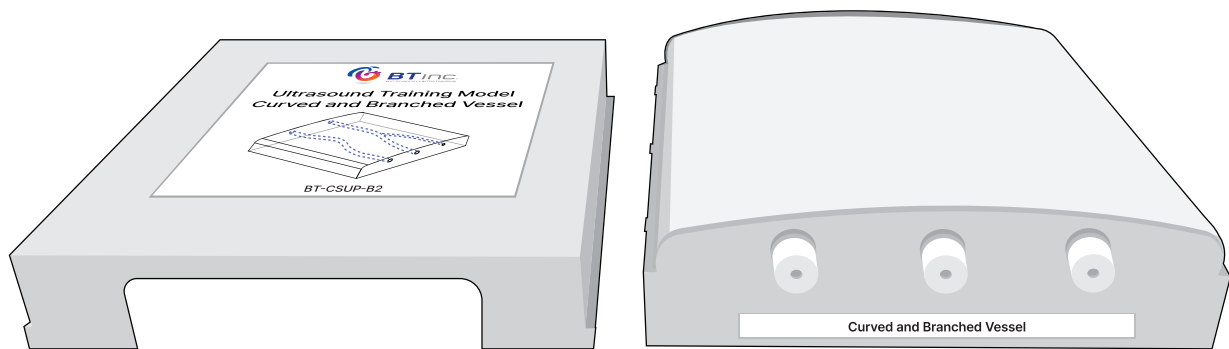


Ultrasound training pad


Recommended Clinical Supply Sizes	
Needle	3cc~5cc, 23G

- The following clinical supply sizes are recommended for use with our Ultrasound training model.
- Other sizes may cause damage to the products.
- Lifetime of the consumables may differ depending on the skill level of the users.



Installation

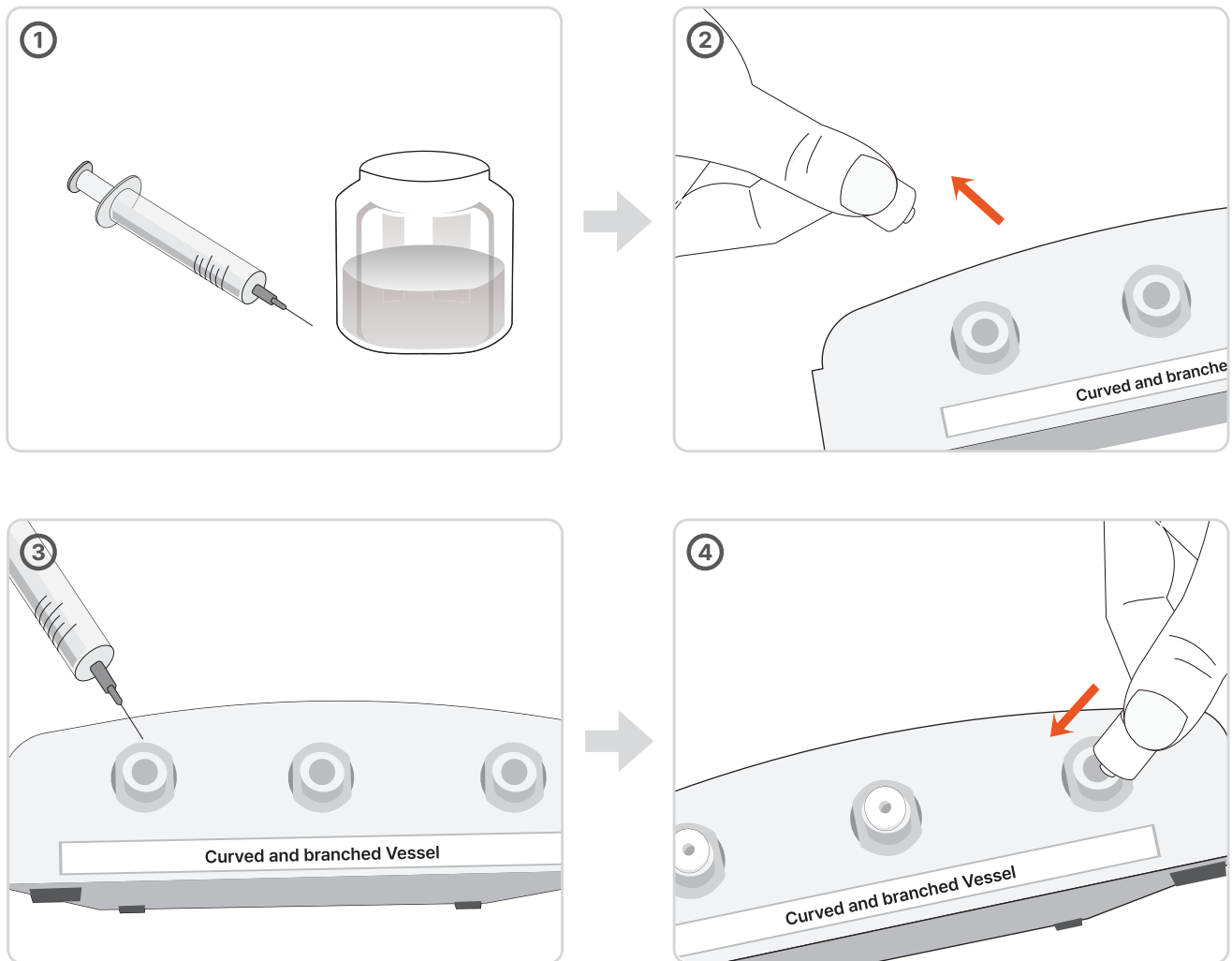

- ① Prepare a syringe and solution (water).
- ② Open the Luer lock caps on the block.
- ③ Using a syringe with a needle attached, inject the solution.
- ④ Close the Luer lock cap and proceed with the ultrasound training.
- * It is recommended to use only clean water for the training solution.

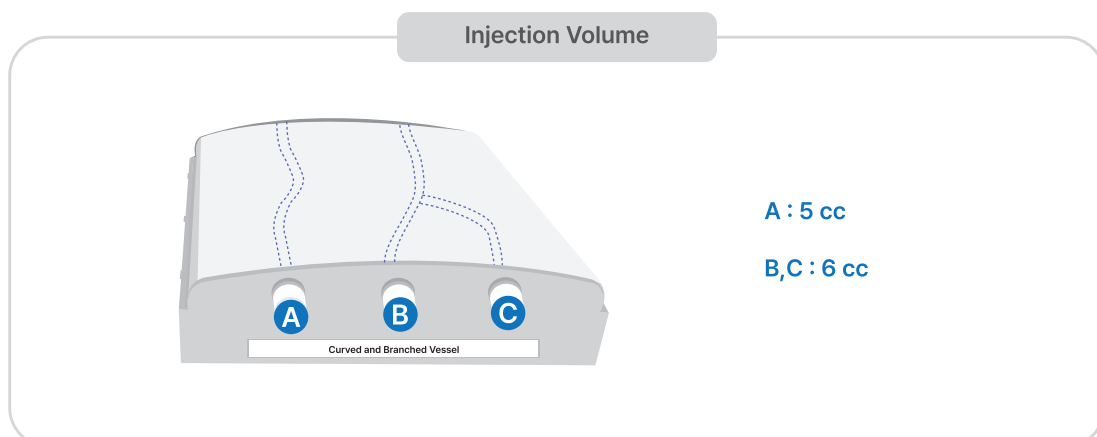

Cleaning and Storage

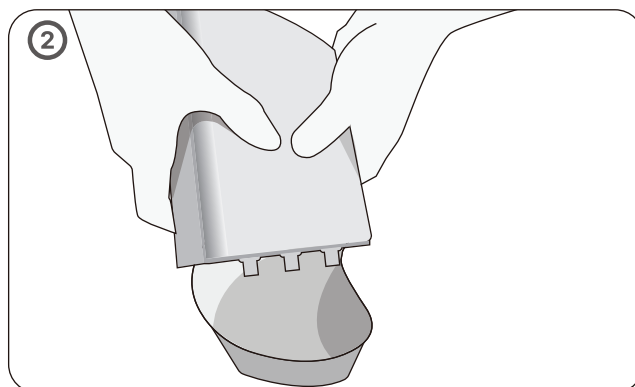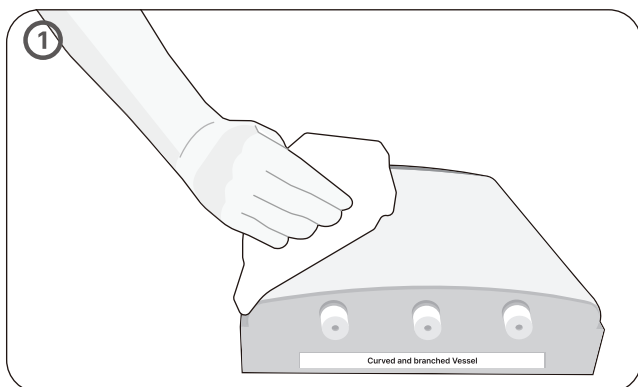

- ① Wipe the exterior of the product clean with an ethanol swab or a soft cloth moistened with ethanol.
- ② For long-term storage, drain the solution from inside the ultrasound training pad before storing.

BT-CSUP-B2

Ultrasound Training Model

A-313, Samsung Techno-valley, 140, Tongil-ro
Deogyang-gu, Goyang-si, Gyeonggi-do, 10594, Republic of Korea

Homepage: www.btinc.co.kr / E-mail: bt@btinc.co.kr
Tel: +82-2-2038-2903, +82-51-903-1144 / Fax: +82 303-3130-0607

Copyright(c) BT Inc. All rights reserved.